






Warm Tips

Please read the instructions of the electrodes carefully before use.

- Do not use it when driving a motor vehicle, bathing, showering, swimming, or exercising.
- Do not use the electrodes on the following areas: heart, spine, eyes, mouth, carotid artery, or throat; broken, excessively swollen and red, infected and inflamed, or rash skin (e.g. phlebitis, thrombophlebitis, varicose veins).
- Do not share the electrodes with others to avoid adverse skin reactions and disease transmission.
- Keep electrodes out of the reach of children or pets.
- It is recommended to attach electrodes to a flat area to ensure that the electrodes do not warp. Do not overlap or bend the electrodes when applying the electrodes to the skin, or the device may stop working, and the gel may be damaged when separating.
- The electrodes should be at least 1 inch apart from each other. Changing the distance between the electrode pieces or the orientation of the electrode pads can change the electrical sensation.
- During use, please do not remove the electrodes and wires to avoid electric shock.
- Attaching the electrodes to one area for a long time will make the skin unbreathable and lead to redness or itching, 20-30 minutes per part are optimal.
- To extend the electrodes' lifespan, drop a few drops of water on the sticky side and dry it naturally with the gel side facing up, then it will become sticky again. (Excessive water will decrease the stickiness.)

Specifications of Electrodes

Material of the Pad	Non-woven Fabric Lead Wire The Conductive Layer Gel		Protection from Sun Damage
Ingredient of the Adhesive	Glycerin non-tallow, Water, Poly(acrylate) copolymer, Latex-free.		Storage Temperature: 5°C-35°C/41°F-95°F
Specification of the Connector	2mm Pin-hole Connector		Be Careful
Package	Individually Sealed		Latex Free
Certification	CE, FDA Cleared		Made of Renewable Materials

EB1520-SMS-V1.0-220722



USER MANUAL


AUVON TENS Unit Electrodes

Resealable Packaging Keeps Electrodes Fresh


Electrodes Placement Suggestion

The followings are electrodes placement suggestions for 3 types of electrodes. Refer to the type of electrodes you purchase to find out where they should be placed.


Square Electrodes

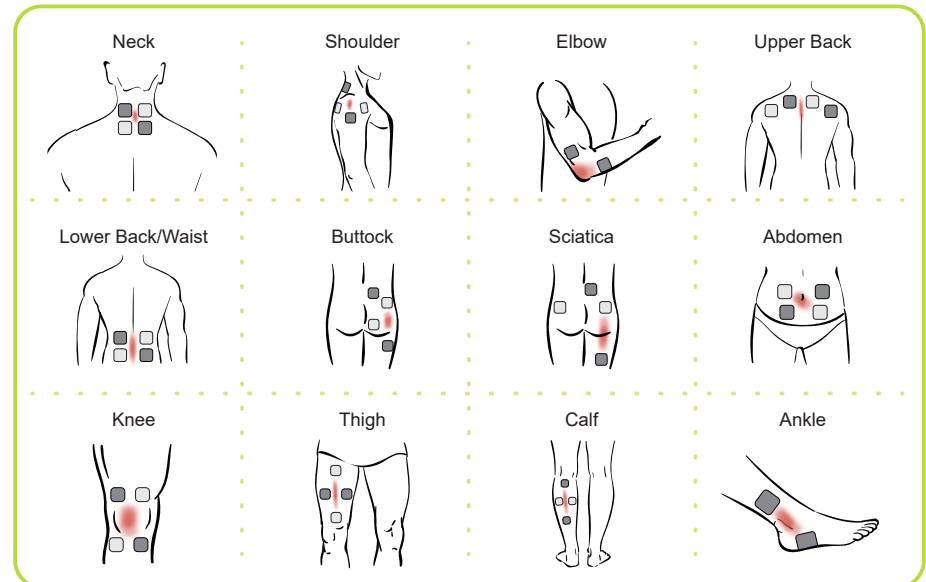

Rectangular Electrodes

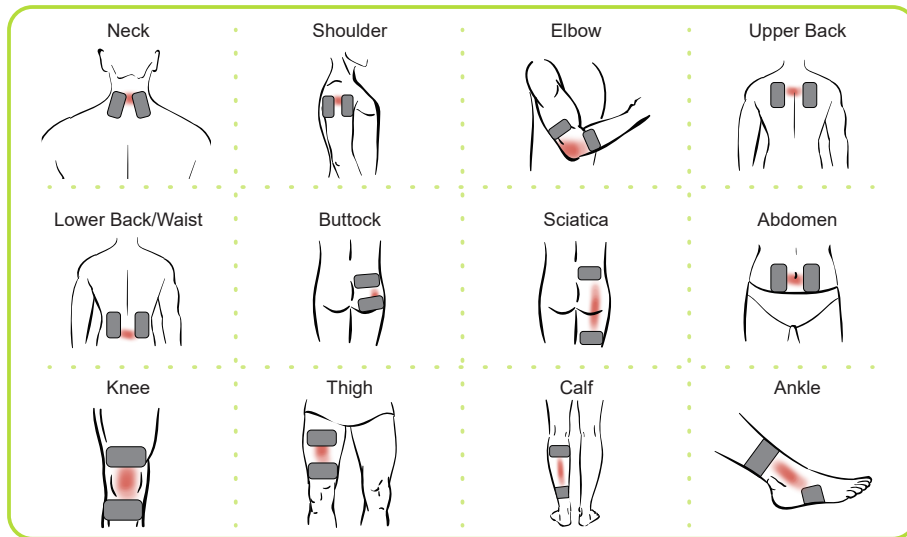

Small Round Electrodes

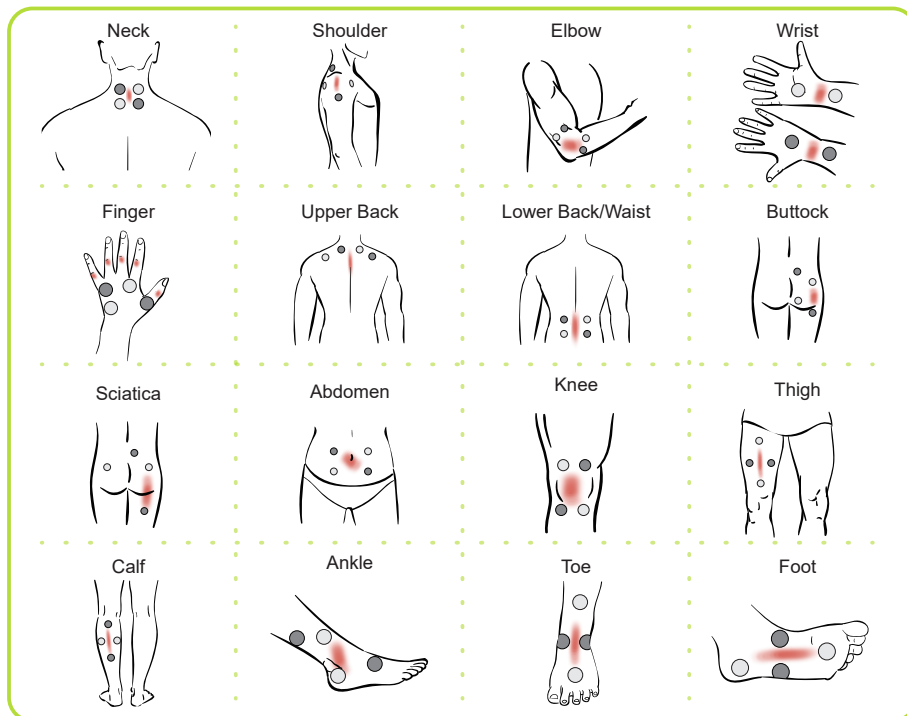

Application and Maintenance of the Electrodes

1



Clean and dry the flat application area (no hair and no wounds).

2




Take out the pads and keep the packaging for future storage.

3



Connect 1 lead wire to a pair of pads and the device port.

4

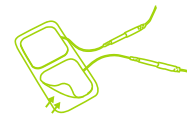

Grab the corner of the pad and peel it off the protective film. **(Do not pull the wire.)**

5



Attach and press the pads firmly on the skin to make sure full contact. (Unit Off)

6



Turn on the unit, and set the mode, intensity, and time (20-30 minutes for each part).

7



After use, turn off the device, grab the corner of the electrodes, and peel it off the skin; **(Do not pull the wire.)**

8



Place the electrodes back into the protective film, remove the wire and seal them in the packaging.

9



Please store in a cool (between 5°C -35°C /41°F -95°F) and dry place.

TIP: Please refer to the electrodes placement suggestions described in this manual for proper electrodes placement according to your electrodes type.