



Aluminium Folding Rollator
(OS-RL91067)

Instruction Manual



** Please read the instruction carefully before using.

Thanks for choosing OasisSpace as your health online store. It will be highly appreciated if you could post your advice, comment on our Page view. If you need any help, kindly contact OasisSpace Customer Service Department.

GENERAL INFORMATION

OasisSpace rollator is designed for both indoors and outdoors.

It is designed exclusively as a walker not a wheelchair. Please no moving while sitting.

⚠ DO NOT use the device if you have any of the following situation:

- Visual impairments
- Inability to be seated
- Severe balance issues
- Disabilities of Arm, Shoulder or Hand

THIS IS A WALKING AID ONLY AND IS NOT TO BE USED TO BE USED AS A TRANSPORTATION DEVICE!!

PRODUCT INFORMATION

OasisSpace rollator is delivered by brown box. After unpacking. Please keep the box for future storage. Increase efficiency and reduce packaging waste.

If the package or product is damaged, please contact us.

** Product Overview



1. Seat Back Belt
2. Mesh Bag
3. Front Wheel Locker
4. Front Wheel
5. Brake Adjustment Screw..
6. Back Wheel
7. Tilt Support
8. Back Wheel Locker
9. Seat Pad with Folding Handle
10. Height Adjustment Locker
11. Brake Handle
12. Armrest Handle

** Product images only for reference purpose, so please check the subject produce with "Package Content"

** Package Content

1. OS-RL91067	x1 set (Red or Blue)
2. Instruction Manual	x1
3. Accessories	x1 Cane Holder x1 Seat Back Belt x1 Mesh Bag
4. Repair Parts	Springs/Screws for Height Adjustment Locker x1 Allen wrench for Brake Adjustment

QUICK USING GUIDE

** INSTALL

Step 1. Install the front wheel

Press the red button, adjust the direction of the wheel, insert and lock it firmly.



Step 2. Install the back wheel

Check the slot of the metal leg and insert to the rollator body. You will hear "CLICK" if you operate well. Then screw up the long handle bolt. Double check it is firmly installed.



Step 3. Push on the seat

Open the rollator and push on the seat from the edges. You will hear "CLICK" if you operate well. Double check it is firmly installed.



Step 4. Install the back belt and mesh bag

Install the belt in this way.



<1> Lower down the handle and put up the brake cable.



<2-2> Insert the buckle to the bolt

<3> Press the buckle to the end

Hang the mesh bag on the seat.



** Prepare to Go

Adjust the height of armrest

Press the red button and adjust the height of armrest to the confirmable position. Make sure both of armrest are in the same height.



Brake and Park

Pull both brake handles upwards to stop the rollator as you need.



Push the brake handles downwards until they lock into place with "click" to park the rollator. Pull the brake handles back up to release parking position.



The seat could be used ONLY in parking position

Tilting Aid

Step on the tilting aid, pull back on the handles at the same time to raise the front wheels.




This will allow you to get over curbs and other small obstacles with little effort.

Please keep the rollator stable when use the tilting aid.

Adjust the brake


- * Loosen allen screw (arrow 1)
- * Tighten or untighten the cable by pulling it out or pushing it in (arrow 2).
- * Then refasten allen screw.

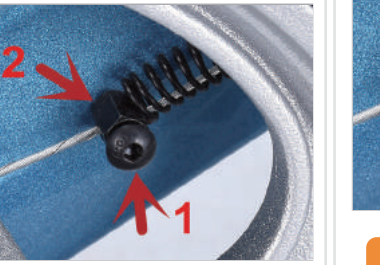

Do not move the rollator when adjust the brake.

Small adjustments:

- * Loosen counter nut (arrow 1) so that the setup screw (arrow 2) can be turned.
- * Turn the setup screw clockwise: to lessen brake effect.
- * Turn the setup screw counterclockwise: to increase brake effect.
- Finally, retighten the counter nut.

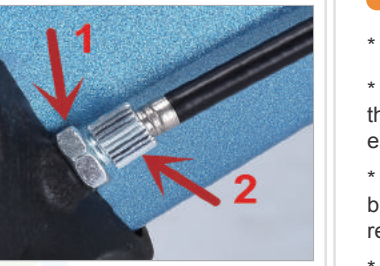

** Storage

To fold and storage the rollator, you just need pull up the handle on the seat and operate the install step oppositely.



**Repair Parts



x3 screws and x3 springs for Height Adjustment Locker Repairing



x1 Allen wrench for Brake Adjustment

TIPS FOR FIRST USE

* Please check the brake, wheels before using.

* The rollator should be used on a level, dry ground. Practice using the rollator until you feel comfortable. Using correctly could help you enjoy your walking!

* When you use the rollator please keep your hands grip the brake. It can ensure you to stop the rollator immediately and effectively reduce the risk of falling.

* Avoid sudden braking otherwise the rollator will slip especially on the wet smooth surface.

* If possible please find somebody to help you to go up stairs, take the elevator and e.g.

* The maximum weight capacity is 300-350 lbs and 11 lbs for mesh bag.

* It is not recommended to use the rollator in BAD WEATHER (Rain/Storm/Lighting e.g.) or on the smooth/wet ground.

* Please slow down when start heading downhill.

CLEAN AND STORAGE

* The frame part can be wiped with a damp cloth or washed with neutral cleaning agent.

* The wheels can be cleaned with a damp brush with plastic bristles (DO NOT use wire brushes)

* The seat back, seat and transporting pouch can be washed with a mild soap.

* Do not use a high pressure or steam cleaner.

* Dry the rollator with a towel. If possible use some machine oil on the parts of brake to avoid rusting such as brake cable, screw, e.g.

PRODUCT SPECIFICATION

Size and Weight	
Seat size	17.7x9.8 in
Seat height	21.2 in
Total width	25.7 in
Total length	27.8 in
Total height/Height of handle (adjustable at 6 levels)	31.89~36.85 in
Empty weight (include belt and mesh bag)	19.1 lb
Rollator weight Capacity	300-350 lb
Mesh bag weight Capacity	11 lb
Folding size	27.8 x 9.8 x 31.89 in
Wheels	
Front wheels	10 x 1.37 in
Back wheels	8 x 1.37 in
Materials	
Frame	Aluminum (corrosion protection)
Seat	Nylon, black
Seat back	Nylon, black
Tire	Polyurethane
Operating Conditions	
Temperature	0 - 122°F
Humidity	20 - 80 %
Storage Conditions	
Temperature	-4 bis + 140 °F
Humidity	45 - 60 %