BEBIRD PRO

USER MANUAL

BEBIRD Pro Cares For You More

Model: T15 PRO

Official Website: bebird-pro.com

CONTENT

1. ENGLISH	03-07
2. Deutsch	08-12
3. Français	13-17
4. Italiano	18-22
5. Español	23-27
6. 日本語	28-32

ENGLISH

Thank you for your purchase on Bebird pro!

We are committed to providing the best products to our customers, and all Bebird pro products meet strict standards for manufacturing, performance, and safety.

We are also committed to providing the best customer service to ensure customer 100% satisfaction, and we offer one year warranty to protect customer rights.

If you need after-sales service or other product consultation, you can contact us in the following ways.

Precautions Before Use

Attention:

1. When cleaning lens, please wipe it carefully with professional alcohol cotton swab.
2. Make sure the ear spoon is assembled in place before using to prevent falling off.
3. Please don't use it while you are moving, and protect yourself from being hit by others.
4. It is forbidden for children to use this product alone to avoid accidental harm.
5. Please do not put the product in liquid to avoid damage.
6. Product built-in polymer battery, so it must be charged once a month if it is not used frequently.
7. Avoid product exposure to sunshine directly, especially the accessories, in order to avoid softening.
8. During using it, the temperature of the product will rise slightly(Data from bebird pro lab, the maximum temperature of the machine doesn't exceed about 45°C), please rest assured to use.


Disclaimer

Bebird pro does not guarantee its goods against any defect or damage caused by transportation, storage, improper installation and maintenance, internal or external hostile environments, misuse, abuse, negligence, accident, modification, tampering, the attachment of unauthorized accessory, alteration to the goods, poor workmanship, or any other conditions whatsoever that cause a defect or failure of the product.

Product Parts Introduction

Brand:	Bebird pro
Product Name:	Smart Visual Ear-Clean Rod
Mode:	T15 PRO
Rod Weight:	25g
Network Standard:	IEEE802.11b/g/n
Antenna:	Internal FPC antenna
Working Frequency:	2.4Ghz
Image Transmission Rate:	20fps
Image Sensor:	CMOS
Working Temperature:	14-113°F
Battery:	300mAh lithium battery
Using time after fully charged:	85 minutes
Charging time:	85 minutes
Input Voltage:	DC 5V 0.5A
Lens Diameter:	3.5mm
Depth of Field:	10~50mm
Pixel:	5M
Gyroscope:	3 axis
Waterproof:	only the lens is waterproof

Product Parts Introduction

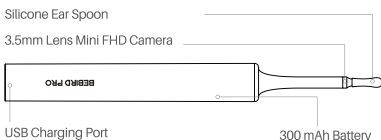EAR PICK ACCESSORIES

Advanced Cleaning Set:

- Silicone covered metal ear spoon(s) *2
- Silicone covered metal ear spoon(l) *1
- Double color ear spoon(s) *2
- Double color ear spoon(l) *2
- Double color ear spoon(r) *2
- Sponge ear spoon *3
- New ear spoon storage bucket(s) *2

Basic Examination Kit:

- Ear canal observation tool *1
- The cover of ear canal observation tool *10

Operation Guidelines

How to connect APP and use it?

1

Scan QR code to download APP.
(you can also download APP from
our official website:
www.bebird-pro.com)

2

Open bebird pro and
select T15 Pro

3

Open the lid of the product and it
will power on automatically, and
let your phone connect product
WiFi "bebirdpro-XXXX"

4

Back to bebird pro and
click on "Start" to use it

Replace ear spoon

Rotate the old ear spoon counterclockwise to
remove the old ear spoon. Align the top of the
cleaning rod with the new ear spoon, then turn
the new ear spoon clockwise to tighten it.
Please make sure the ear spoon is assembled
in place before using it to prevent falling off.

Truoble Shooting

Question 1: The product body heats up during use

Answer: The product lens is equipped with an LED light, which is not dazzling or glare. And the product adopts a dual temperature control system. During use, the otoscope part has a non-sensing temperature of 25° and the body part's palm temperature is 32°. Therefore, It is a normal phenomenon that the product occurs with slight heat during the operation. The product is safe, so you can use it with confidence.

Question 2: APP display abnormally (lens blur/fuzzy/black screen)

Answer: Uninstall the APP and reinstall, ,confirm that the right machine model has been choosed on APP. Wipe the lens of the product with an dry swab to check whether the lens is damaged.

Question 3: Why does the lens blur after washing? (Propaganda ip67 waterproof)

Answer: Waterproof IP67 refers to splash-proof. It is not recommended to wash with water directly. If the lens enters the water, . Please note that only the lens is waterproof.

Question 4: The WiFi connection is unstable, and the connection will be automatically disconnected from time to time

Answer: The WiFi of products is limited to the use of the product and cannot access the Internet. When It prompts connected but unable to access the Internet, whether to switch, please click "No"; if the WiFi coverage area is strong and the signal of the product is affected by the WiFi coverage, it is recommended to change the environment and try.

Question 5: After using it for a few months, the APP shows that it is not connected after connecting to WiFi (there is no problem with the previous connection)

Answer: Re-download the APP and try to connect and use it.

Warranty

All Bebird pro products are under Bebird pro's warranty for **ONE Year** after the date of the original purchase. Bebird pro recommends using genuine products and accessories purchased from our brand for optimal performance. Using other brush heads may damage your ear scoop and limit your warranty coverage. Contact our after-sales email: cs@bebird-pro.com

Warranty Exclusions

What is not covered by the warranty?

- Use of unauthorized parts or wrong use of the ear cleaning tool.
- Use of the brush head due to misuse, abuse, neglect, alteration, or unauthorized repairs.
- Normal wear and tear, including debris, scratches, scuffs discoloration or fading.

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