
CUREGO Note 3 PRO MAX

USER MANUAL

Creativity For Your Health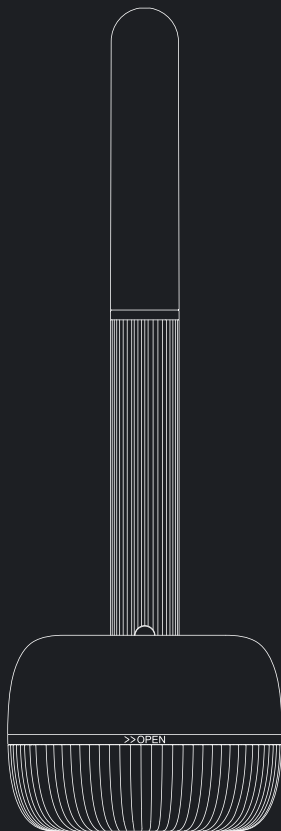

 @curego  @curego  @curego
 cs@curegos.com  curego

CONTENT


1. ENGLISH -----	03-07
2. Deutsch -----	08-12
3. Français -----	13-17
4. Italiano -----	18-22
5. Español -----	23-27
6. 日本語 -----	28-32

Dear Customers :

Thank you for purchasing an CUREGO product. Please read this user manual carefully and keep it for future reference. If you need any assistance, please contact our Email: cs@curegos.com with your product model number and Amazon/Website order number. We will make every effort possible to ensure your satisfaction with your purchase as well as resolve any issues you run into. In addition, we are looking forward to your valuable opinions about our products. Every comment will be valued by CUREGO.

Package Contents


Note 3 Pro Max*1

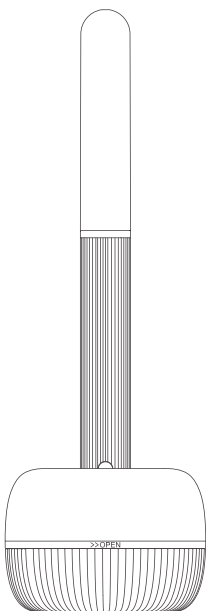Silicone ear spoon(in the base)*12


Alcohol swab *5

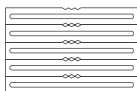Softened swab *5

Small brush*1 Protective plug*1

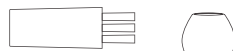Type-C Cable*1

OPERATING GUIDE

How to Connect APP and Use it?


- 1 Scan QR code to download APP.(you can also download APP from our official website: curegos.com)

- 2 Open curego APP, select curego Note 3, then open the product cover and it will power on automatically


- 3 Connect product WiFi "curego-XXXX"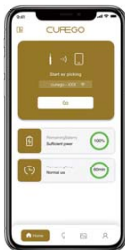

- 4 Touch the start button to use it

How to replace ear spoon?

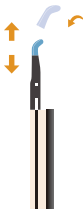Unscrew the fixed ring and pull out the old ear spoon. Install the new ear spoon and tighten the fixed ring. Please make sure the ear spoon is assembled in place before using to prevent falling off.

IMPORTANT SAFETY INSTRUCTIONS

Pre-use Notice :

1. Please make sure this product was purchased from a legal sales channel.
2. This product may contain small parts, and we strongly advise you to keep this product away from infants and children, so as to prevent a choking hazard or possible injury.
3. Before using, please make sure the product hasn't been damaged or broken, or fallen to the ground. If there are any of these conditions, please check to ensure that it doesn't affect the use.
4. This product is recommended to be stored in a cool and dry place. Keep away from moisture and dust, otherwise it may be damaged.
5. Keep the product away from hazardous areas or materials exposed to flames or easy to burn.

Disclaimer :

CUREGO does not guarantee its goods against any defect or damage caused by transportation, storage, improper installation and maintenance, internal or external hostile environments, misuse, abuse, negligence, accident, modification, tampering, the attachment of any unauthorized accessory, alteration to the goods, poor workmanship, or any other conditions whatsoever that cause a defect or failure of the product.

CUREGO's sole responsibility under this warranty shall be, at its option, to either repair or replace any goods which fail during the warranty period, provided that the buyer has promptly reported the said failure to CUREGO in writing, and complies with the provisions of this warranty.

PRODUCT SPECIFICATION

Brand :	CUREGO
Product Name :	Smart Visual Earwax Cleaning Tool
Modell :	Note 3 Pro Max
Rod Weight :	128g
Network Standard :	IEEE802.11b/g/n
Antenna :	Internal FPC antenna
Working Frequency :	2.4Ghz
Image Transmission Rate :	20fps
Image Sensor :	CMOS
Working Temperature :	14-113°F
Battery :	300mAh lithium battery
Using time after fully charged :	70 minutes
Charging time :	60 minutes
Input Voltage :	DC 5V 0.5A

Lens Diameter :	3.5mm
Depth of Field :	15 ~ 20mm
Pixel :	10 million pixels
Gyroscope :	360° Gyroscope

TROUBLE SHOOTING

Question 1 : The product body heats up during use

Answer : The product lens is equipped with an LED light, which is not dazzling or glare. And the product adopts a dual temperature control system. During use, the otoscope part has a non-sensing temperature of 25° and the body part's palm temperature is 32°. Therefore, It is a normal phenomenon that the product occurs with slight heat during the operation. The product is safe, so you can use it with confidence

Question 2 : Abnormal connection

Answer : Confirm whether the machine is turned on in a normal state (the ring indicator flashes when not connected, and the indicator light will be always on after successful connection)

Question 3 : APP display abnormally (lens blur/fuzzy/black screen)

Answer : Uninstall the APP and reinstall, confirm that the right machine model has been choosed on APP . Wipe the lens of the product with an dry swab to check whether the lens is damaged. (Not advise alcohol swab)

Question 4 : Abnormal charging: After using for a period of time (months/half a year), it can't be charged or can't be turned on after charging

A : Wireless charging by magnetic base: place the device on a horizontal table. If the indicator light flashes abnormally while charging, please turn the product body to check whether it can be charged normally. If the contact is poor and usage has been affected, please contact customer service.

Question 5 : The WiFi connection is unstable, and the connection will be automatically disconnected from time to time

A : The WiFi of products is limited to the use of the product and cannot access the Internet. When it prompts connected but unable to access the Internet, whether to switch, please click "NO"; if the WiFi coverage area is strong and the signal of the product is affected by the WiFi coverage, it is recommended to change the environment and try.

Question 6 : After using it for a period of time, the indicator light will go out soon when charging, and it will not turn on.

Answer : Try to turn the product or contact again to check whether the charging contact is bad.

Question 7 : After using it for a few months, the APP shows that it is not connected after connecting to WiFi (there is no problem with the previous connection)

Answer : Re-download the APP and try to connect and use it.

Warranty

All CUREGO products are under CUREGO's warranty for **ONE Year** after the date of the original purchase. CUREGO recommends using genuine products and accessories purchased from our brand for optimal performance. Using other brush heads may damage your ear scoop and limit your warranty coverage. Contact our after-sales email: cs@curegos.com

WARRANTY EXCLUSIONS

What is not covered by the warranty?

- Use of unauthorized replacement parts or Unauthorized use of the ear scoop.
- Use of the brush head due to misuse, abuse, neglect, alteration, or unauthorized repairs.
- Normal wear and tear, including debris, scratches, scuffs discoloration or fading.

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