XLIFE

e: www.xlifecare.com

Email Address: cs@xlifecare.com

Tiktok: [@Xlifeofficial](https://www.tiktok.com/@Xlifeofficial)

Skype: [+1\(800\)484-0309](https://www.skype.com/people/+1(800)484-0309)

Facebook: [@Xlifeofficial](https://www.facebook.com/Xlifeofficial)

Youtube: [Xlife](https://www.youtube.com/Xlife)

XLIFE

LIFE IS MORE EXPERIENCE

User Manual

X3

INTRODUCTION

Dear customers:

Thank you for purchasing our XLIFE product. This introduction will help you to properly use your new earwax removal tool. Please read these carefully before using and keep them for future reference. If you need any other assistance, please contact our customer services with the model of the product and order number from the following channels:

Email Address: cs@xlifecare.com

Tiktok: @Xlifeofficial

Skype: +1(800)484-0309

Facebook: @Xlifeofficial

Youtube: Xlife

Official Website: www.xlifecare.com

We will make every effort to ensure your 100% satisfaction with your purchase as well as resolve any issues you run into. In addition, we are looking forward to your valuable opinions about our products. Every comment will be valued by XLIFE.

CONTENT

1. ENGLISH	1-7
2. Deutsch	8-13
3. Français	14-18
4. Italiano	19-23
5. Español	24-28
6. 日语	29-35

PRECAUTIONS BEFORE USE

Attention:


1. When cleaning lens, please wipe it carefully with professional alcohol cotton swab.
2. Make sure the ear spoon is assembled in place before using to prevent falling off.
3. Please don't use it while you are moving, and protect yourself from being hit by others.
4. It is forbidden for children to use this product alone to avoid accidental harm.
5. Please do not put the product in liquid to avoid damage.
6. Product built-in polymer battery, so it must be charged once a month if it is not used frequently.
7. Avoid product exposure to sunshine directly, especially the accessories, in order to avoid softening.
8. During using it, the temperature of the product will rise slightly (Data from XLIFE lab, the maximum temperature of the machine doesn't exceed about 45°C), please rest assured to use.

Disclaimer

XLIFE does not guarantee its goods against any defect or damage caused by transportation, storage, improper installation and maintenance, internal or external hostile environments, misuse, abuse, negligence accident, modification, tampering, the attachment of unauthorized accessory, alteration to the goods, poor workmanship, or any other conditions whatsoever that cause a defect or failure of the product.

PACKAGE CONTENTS

1 x Xlife X3 Smart Visual Earwax Cleaning Tool
1 x silicone covered metal ear spoon
1 x Micro-USB Charging Cable
1 X User Manual

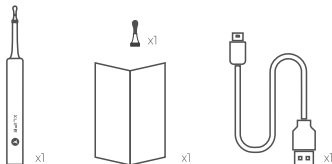PRODUCT SPECIFICATION

Brand: XLIFE

Product Name: Smart Visual Earwax Cleaning Tool

Model: X3

Rod Weight: 13g

Network Standard: IEEE802.11b/g/n

Antenna: Internal FPC antenna

Working Frequency: 2.4Ghz

Image Transmission Rate: 20fps

Image Sensor: CMOS

Working Temperature: 14~113°F

Battery: 130mAh lithium battery

Using time after fully charged: 45 minutes

Charging time: 45 minutes

Input Voltage: DC 5V 0.4A

Lens Diameter: 3.5mm

Depth of Field: 10~50mm

Pixel: 3 million

Gyroscope: 3 axis

Waterproof: No

OPERATION GUIDELINES

How to connect APP and use it?

1. Scan QR code to download APP. (You can also download APP from our official website: www.XLIFE.com)
2. Open XLIFE and select X3.
3. Long Press the power button to turn on the ear cleaning rod, and let your phone connect product WiFi "XLIFE-XXXX"
4. Back to XLIFE and start to use it.

1

2


③

④

How to replace ear spoon?

Rotate the old ear spoon counterclockwise to remove the old ear spoon. Align the top of the cleaning rod with the new ear spoon, then turn the new ear spoon clockwise to tighten it. Please make sure the ear spoon is assembled in place before using it to prevent falling off.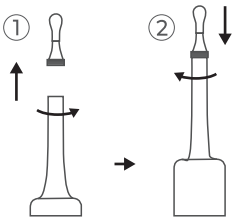

TRUEBLE SHOOTING

Question 1: The product body heats up during use

Answer: The product lens is equipped with an LED light, which is not dazzling or glare. And the product adopts a dual temperature control system. During use, the otoscope part has a non-sensing temperature of 25° and the body part's palm temperature is 32°. Therefore, it is a normal phenomenon that the product occurs with slight heat during the operation. The product is safe, so you can use it with confidence.

Question 2: Abnormal connection

Answer: Confirm whether the machine is turned on in a normal state (the ring indicator flashes when not connected, and the indicator light will be always on after successful connection).

Question 3: APP display abnormally (lens blur/fuzzy/black screen)

Answer: Uninstall the APP and reinstall, confirm that the right machine model has been chosen on APP. Wipe the lens of the product with an dry swab to check whether the lens is damaged. (Not advise alcohol swab)

Question 4: Abnormal charging: After using for a period of time (months/half a year), it can't be charged or can't be turned on after charging.

Answer: Charging by USB/Type-C cable: After connecting to the power supply, the indicator light always lights up when the charging process is normal. If the indicator light flashes during the charging process or the indicator light does not light up, please replace the charger or check whether the device is correctly connected to the power supply.

Question 5: The WiFi connection is unstable, and the connection will be automatically disconnected from time to time

A: The WiFi of products is limited to the use of the product and cannot access the Internet. When it prompts connected but unable to access the Internet, whether to switch, please click "NO"; if the WiFi coverage area is strong and the signal of the product is affected by the WiFi coverage, it is recommended to change the environment and try.

Question 6: After using it for a period of time, the indicator light will go out soon when charging, and it will not turn on.
Answer: Try to change the data cable or change the charging head to direct USB charging, then turn the product or contact again to check whether the charging contact is bad.

Question 7: After using it for a few months, the APP shows that it is not connected after connecting to WiFi (there is no problem with the previous connection)
Answer: Re-download the APP and try to connect and use it.

WARRANTY

All XLIFE products are under XLIFE's warranty for ONE Year after the date of the original purchase. And we are committed to providing the best products as well as the best customer service to our customers. If you need any assistance, please contact our customer service with your product model number and order number. We will resolve any issues you run into as soon as possible.

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