



Blood Glucose Monitoring System EBG-100SL

Q1: Can I use it with someone else, like my husband?

No, the Easy@Home Blood Glucose Monitoring System is intended to be used by a single person and should not be shared.

Q2: Can I change my medication based on the test results from the Easy@Home Blood Glucose Monitoring System?

No, it is important not to change medication based on the test results without the advice and consultation of a physician or healthcare professional.

Q3: How long is the service life of the lancing device?

The service life of the lancing device is no less than 3000 times.

Q4: Can I reuse a test strip if I didn't apply enough blood the first time?

No, it is not recommended to reuse a test strip if you didn't apply enough blood. If the amount of blood applied was too small, an error code "E42" will appear, indicating that the blood is not enough. To repeat the test, please use a new test strip with a larger blood drop.

Q5: How long can I use the test strips after opening the vial?

It is recommended to use the test strips within 90 days after first opening the vial cap. Remember to record the discard date on the vial when you open a new vial of test strips.

Q6: Can I transfer the test strips to a new vial for convenience?

No, it is not recommended to transfer the test strips to a new vial. Always carry the test strips in their original vial to ensure their integrity and accuracy.

We hope these tips were useful. If you have any inquiries about our products or services feel free to contact us via email, call our customer service at 855.822.6999 M-F 9am-5pm CT or visit us at [healthcare-manager.com](https://www.healthcare-manager.com)