



# VOCiC ROLLATOR

INSTRUCTION MANUAL

>>> English 1



- V-RL-Z71-A-RD RED ROLLATOR
- V-RL-Z71-A-BK BLACK ROLLATOR
- V-RL-Z71-A-GN GREEN ROLLATOR

✉ [service-us@vocic.vip](mailto:service-us@vocic.vip)

# Content

I.Safety Instruction.....	1
II.Product Information.....	2
III.Product Overview.....	3
IV.Assembly Instructions.....	3
V.Operation Instruction.....	4
VI.After-sales Service.....	6

**Thank you for purchasing a VOCIC Rollator!**  
**The VOCIC Rollator made of aluminium has been developed with a lot of attention to safety and will support you in your everyday life.**

Please read these directions carefully before you start using the product and keep them for future reference. If you have any other questions or valuable suggestions, please send to: [service-us@vocic.vip](mailto:service-us@vocic.vip), we look forward to hearing from you, thank you.

Manufacturer: Zhongshan ANJ Medical Equipment Co.,LTD

Address: NO.3 of Shenghui South Road,Nantou Town, Zhongshan City,Guangdong, China

Distributor: Hengshui Ruilangde Medical Equipment Co., Ltd

Address: East D block of Haiwei West Road,North of Zhongkuang Road MiaozenDevelopment Zone, Jingxian County, Hengshui City, Hebei Province

## I. Safety Instructions

### ■ Safety Precautions Before Use

- Make sure the rollator is completely unfolded and locked into position.
- Check that the rollator is fully intact each time before using it. Check that the brake is working. Check that the backrest is fixed properly.
- Use only on even and solid surfaces, never on an inclined or uneven ground.

### ■ Accident Prevention

- Always use both brakes at the same time! Do not sit on the rollator if the parking brake is not active or if it is at an angle.
- The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in a fall.
- All four wheels must be in contact with the ground at all times. Only in this way the rollator is safely balanced which prevents accidents.
- Always be careful when crossing thresholds, railway tracks, gully covers with wide slits and the like. A wheel can get caught or trapped in them, stop abruptly and you might fall.
- Do not use the Rollator if any part has been damaged, broken or cracked, or if it is missing.
- Regularly check the screws and fasteners on the Rollator and retighten if necessary.
- Never use in the presence of flammable/combustible and burning substances.

### ■ Restrictions on Use

- Use only as a walking aid! DO NOT MOVE ROLLATOR WHILE SEATED. This is a walking aid only and is not to be used as a transportation device or a wheelchair.
- The net bag may be loaded with a maximum of 5kg/11Lbs and do not overload it.
- Do not exceed the maximum user weight: 110kg/242Lbs.

### ■ WARNING



Keep the plastic packaging materials away from children due to the risk of suffocation.



## II. Product Information

### ■ Product: Rollator

### ■ Indication for Use

The VOCIC Rollator was developed for indoor and outdoor use. It serves as a mobility aid with four wheels for people with walking difficulties. The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

### ■ Please Note:

We are only able to issue a guarantee if the product is used in accordance with the specified conditions and for the specified purpose.

### ■ Contraindication

The VOCIC Rollator is not suitable for people with very weak arm strength, significant cognitive impairment, or very poor balance. The use and application of the VOCIC Rollator should be assessed and instructed by a doctor or therapist.

### ■ Technical Data

Max. user weight	242lbs	Handle height ( 6 regulations)	32.5-37.4"
Total weight approx.	17.2lbs	Seat (width x depth)	18.1*8.7"
Max. bag load	11lbs	Seat height	20"
Length/Overall depth	28.3"	Front wheels (diameter x width)	9.4*1.4"
Width (open)	23.6"	Rear wheels (diameter x width)	8*1.4"
Width (folded)	9.6"	Measurements folded	28.3*9.6*32.5"
Material of frame	Aluminum	Material of handles	TPE
Material of tyres	PP+PU	Material of seat	PU

### III. Product Overview



- |                                   |                                |
|-----------------------------------|--------------------------------|
| 1. Handle                         | 10. Curb assistant             |
| 2. Brake handle                   | 11. Brake adjustment           |
| 3. Backrest                       | 12. Front wheels               |
| 4. Aluminum frame                 | 13. Rear wheels                |
| 5. Interlock button for handgrips | 14. Rear reflector on handgrip |
| 6. Removable bag                  | 15. Brake shoe                 |
| 7. Folding handle                 | 16. Hand screw                 |
| 8. Seat                           | 17. Silver clip                |
| 9. closure clip                   |                                |

#### Inspection on Delivery

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service ([service-us@vocic.vip](mailto:service-us@vocic.vip)).

### IV. Assembly Instructions

Our rollator is delivered pre-assembled.

## V. Operation Instructions

### 1. Folding System

#### • Unfolding



① Unclip the grey closure clip (9) first.



② Grab both handles, pull outwards until the frame is half-open.



③ Place both hands on each side of the seat and push downwards until you hear a click.

#### • Folding

Lift the seat by the folding handle(7) and push the handles toward each other.

### 2. Height Adjustment

The handles should be level with your wrist when standing in an upright position. If in doubt, consult your physical or occupational therapist.



① Push the button



② Adjust the handle



③ Release



Check that the handle is locked in a secure position.

### 3. Brake System



① To apply the handbrake pull the brake lever up.




② To engage the parking brake, press the brake lever down until it locks.

#### 4. Backrest Adjustment



① To adjust the height of the backrest, loosen the hand screws(16), move to the desired height and retighten the screws(16)

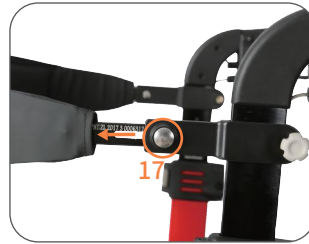

② Press the silver clip(17) whilst simultaneously pulling the end of the backrest to adjust.



The fasteners for the backrest should be at the same height on both sides.

#### 5. Sitting

The Rollator gives support when walking. If you want to take a break, you can sit on it. Place the rollator on a solid, level surface. Lock the brakes (parking position). The Rollator must stand and must not slip. Sit straight up.

#### 6. Using the Curb Aid

Carefully press on the curb aid with a foot. Pull slightly by the handles to lift the front wheels off the ground.



Be aware that the stability of the Rollator changes when it is reclined. Be cautious and try to bypass rather than climb high obstacles.

#### 7. Removing/Attaching the Carry Bag

- To remove the bag, twist knobs located on the front end of the seat frame outward. Pull out knobs and slide bag out of receptacle.
- To replace the bag, fold the Rollator slightly and reverse above steps.

#### 8. Transport and Storage

If the rollator is to be stored or shipped, it is advisable to retain the original packaging. Your rollator will have an optimum protection against environmental influences during storage and transport.

#### 9. Maintenance

- Clean regularly by hand with a damp cloth. Do not use aggressive or caustic detergents.
- Avoid long-term exposure to the sun and rain.
- Do not scratch the surface of the rollator with hard tools, sharp knives, etc.
- If corrosive liquid is splashed on the surface of the rollator, please wipe it quickly with a dry towel.



- If the wheels are found to have sharp and harsh sounds, they should be lubricated regularly.

## VI. After-sales Service

### 1. Replacing of Parts

Wearable parts such as tires are subject to wear and tear. Please contact us for the replacement of parts if necessary.

### 2. Warranty

Once you purchase with VOCIC, you are covered by an 1-year warranty and lifetime after-sales service. Our 24/7 after-sales service is always available to answer any questions you may have during installation and use.

#### Contact

If you have any questions, please contact us: [service-us@vocic.vip](mailto:service-us@vocic.vip)





# Your Home Rehabilitation Partner



Instagram



Facebook



Youtube