


Double Electric Breast Pump

Chef
Handy

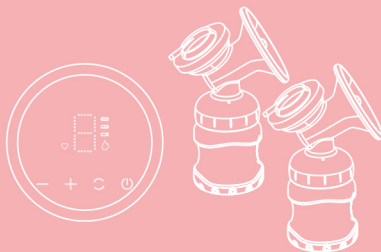

USER'S MANUAL

Model:BBP-8300ED

Product Features

Chefhandy electric breast pump adopts stimulation lactation and normal Suckling mode to simulate the baby's sucking, which is designed for the mother's breastfeeding.



Professional lactation and milking to relieve the pain of milk swelling



5V energy saving, USB multiple power supply mode




Soft suction, rapid frequency stimulation lactation




Equipped with a direct suction breast milk storage bag for convenient milk storage

Description of product components and features

Please check the product parts before use. If there is any missing parts, please contact the dealer.(Note: the pictures are for reference only)

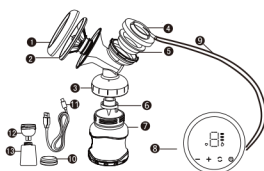

Single

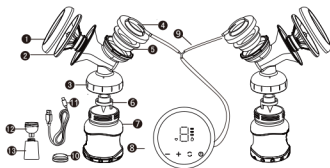

Double


No.	Part Name	Material	Heat resistance temperature	Quantity	
				Single	Double
1	Anti Dust cover	PP	-20~120°C/ -4~248°F	1	2
2	Massage petals	Silicone	-20~200°C/ -4~392°F	1x24mm 1x27mm	2x24mm 2x27mm
3	Electric suction tee	PP	-20~120°C/ -4~248°F	1	2
4	Suction strap cap	PP	-20~120°C/ -4~248°F	1	2
5	Silicone suction strap	Silicone	-20~200°C/ -4~392°F	1	2
6	Duckbill Valve	Silicone	-20~200°C/ -4~392°F	2	4
7	Milk Bottle	PP	-20~120°C/ -4~248°F	1	2
8	Main Engine	ABS	-20~80°C/ -4~176°F	1	1
9	Air pipe	Silicone	-20~200°C/ -4~392°F	1	1
10	Bottle Cap	PP	-20~120°C/ -4~248°F	1	2
11	USB power cord			1	1
Part Name					
12	Storage bag adapter	PP	-20~120°C/ -4~248°F		
13	Storage bag	PET/PE	-20~120°C/ -4~248°F		

The assembly method is as follows


(note: the picture is for reference only, please refer to the real product)



- ◆ Place the massaging petal ② into suction tee ③. Pay attention to covering the edge of the suction tee with the groove of the massage petal to ensure a close match.

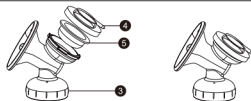

- ◆ First, put silica gel suction strap ring ⑤ into suction tee ③. Pay attention to covering suction tee edge with the edge groove of silica gel suction strap ring. And then press the suction strap cap ④ onto suction tee. As the picture shown.

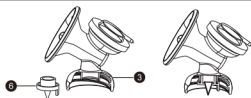

- ◆ Press the duckbill valve ⑥ on the bottom of the suction tee ③ as shown. (Note: Please carefully hold the duckbill valve to prevent damage. Do not use the damaged duckbill valve. Please replace it with a new one.)






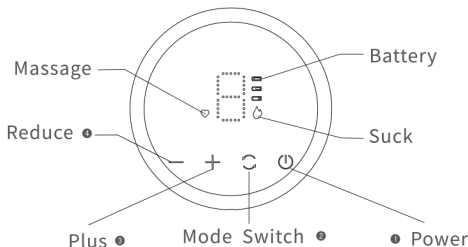
- ◆ Screw the wide-mouthed milk bottle ⑦ into the bottom of suction tee ③, and finally connect the air pipe ⑨ with the main engine ⑧ as shown in the figure. Be sure to insert tightly to prevent air leakage.



- ◆ In order to facilitate storage and suck the milk directly into the breast milk storage bag, Screw the storage bag adapter ⑫ into the storage bag and then screw them together into the bottom of suction tee ③. Finally connect the air pipe ⑨ with the main engine ⑧ as shown in the picture and then you can use it.

Operation instructions



1. The machine has 4 keys in total: ① on-off key  ② lactation and suction mode switch key  ③ gears plus key  ④ gears reduction key .





2. Please clean your hands before use and make sure the breast pump components are thoroughly cleaned and properly assembled.

3. The user maintains a normal and relaxing sitting position. Place a footstool under the foot during breast sucking to help maintain a sitting position and avoid pressing on the waist. (for better effect, warm towel can be used to hot compress and massage the breast before breast suction,)

4. Aim the assembled suction tee at the nipple and hold it tightly against the breast with the other hand to ensure no air enters.

5. Press the "" key on the main engine to automatically enter the buffer breast collection mode, The galactagogue light is on "". The massage stage has a small suction so as to massage the breast and stimulate the breast. Press "+", "-" in the process of use to adjust the selection of appropriate gear, The lactation mode has 5 strength gears in total.

6. Press the mode switch key "" on the main engine. The breast pump light is on "", it enters the normal breast suction mode. There are 9 suction strength gears in this mode. During usage you can touch "+" and "-" for several times to adjust and select the right gear and lactation strength.

- ◆How to find the most suitable suction for you personally? Please gradually increase the suction until you feel slightly uncomfortable and then slightly reduce the suction.
- ◆Recommended to use the routine "suction" function directly after the "lactation" mode.
- ◆The breast pump automatically shuts down after default working time of 30 minutes.

Caution

- ◆Please read the instructions carefully before using it.
- ◆Do not put the main engine into the water for washing. Do not put it into the steam sterilizer for disinfection.
- ◆The connecting tube can be used without disinfection. Please be careful not to twist the knot during use.
- ◆In order to avoid danger, do not place the product in the place that infants can touch.
- ◆Use this product in less lactation stage, please follow the guidance of doctors and nurses.
- ◆If the main engine of this product is damaged, do not disassemble it by yourself. It should be repaired or replaced by the manufacturer or professional person.
- ◆This product belongs to personal exclusive articles. Do not use it by many people for the sake of hygiene and health.

Cleaning & Maintenance

- ◆Please disassemble all accessories that come into contact with your breasts or breast milk, such as bottles, pacifiers, suction tees, massage petals, suction rings, and then clean
- ◆Please clean above accessories immediately after use to avoid the residual bacteria in the milk. Please keep it in a cool and ventilated place after cleaning.
- ◆When disassembling or assembling, make sure that each component is assembled in the assembly sequence.

Fault Resolution

- ◆Failure Phenomenon :The appliance can not work

Solution: Check the battery power, replace the power adapter, make sure that the adapter interface is inserted into the host's DC interface correctly.

- ◆Failure Phenomenon : Insufficient suction

Solution:1.Replace the parts of the duckbill valve, check whether the duckbill valve is installed correctly or not, usually put the duckbill valve in boiling water for 3 minutes and re-install it, then the suction can be restored.

2.Battery power is insufficient. Please charge it in time.

3.Check if the breast cover and the breast are completely fitted and there is no air leakage.

4.If the room temperature is below 15° C/59°F, please continue to run for 30 seconds before breast-sucking operation.

Storage/heating of breast milk

- ◆ After the breast milk is absorbed, the collected breast milk is sealed in a storage bag or other container and then stored in a refrigerator.

	Room temperature	Refrigerator compartment	Refrigerator freezer
Fresh breast milk	Do not place at room temperature	Store for 3-5 days at 4 °C /39°F	Store at -16 °C /3°F for 6 months
Thawed breast milk	Do not store	10hours	Do not freeze again

Note: This Table is only for healthy full-term infants.

Note:

- ◆ When freezing, in order to keep space for liquid expansion, do not fill milk more than 3/4 of the total capacity of the container.
- ◆ In order to ensure that the nutrients are not lost, the breast milk should be thawed in the refrigerator's freezer or thawed in warm water at around 37 °C /98°F.
- ◆ Do not heat frozen breast milk in microwave oven or boiling water to avoid loss of milk nutrition and scalding baby.
- ◆ Shake the milk storage container gently before drinking so that the milk can be evenly mixed.

Product specification

Product Name	Double Electric Breast Pump	Model	BBP-8300ED
Power Supply	DC 5V 2A	Warranty Period	1year



More product? Visit
<http://www.chefhandy.net>



Any issue? Contact us
<http://www.chefhandy.net>

Need Additional Customer Support?
Contact US expert via
email: **Support@chefhandy.com**

Do not return this product
to the place of purchase.

