VELPEAU



[Product Name] : Adjustable Neck Brace

[Product Code]:

(Regular Version) □VP0207 (Inflatable Version) □VP0208

[Product Compounds]:

Modal fibe, Spandex, Artificial Cotton, Mercerized Cotton, Polyurethane, Nylon, TPU, ABS+PC, POM.

【Product Features 】:

•Minimalist style, hollow open design, all-around ventilation for a sweat-free experience.

• Five adjustable height settings for the lower jaw support, conforming to the natural curve of the neck, effectively supporting the head and neck, and relieving pressure on the cervical spine.

•Three adjustable height settings for the rear neck panel support, making the product suitable for different neck heights.

· Regular Version:

Super lightweight, the product weighs only 0.44lb, approximately equivalent to the weight of 4 eggs, allowing for extended wear without burdening the neck and shoulders, and without hindering daily activities.

- 1. Super lightweight, the product weighs only 0.66lb, approximately equivalent to the weight of 6 eggs, allowing for extended wear without burdening the neck and shoulders, and without hindering daily activities. 2,Inflating and deflating is easy and convenient. After wearing, you can adjust the inflation to achieve a more comfortable and stable fit.
- •Made from aerospace-grade engineering plastic material with high-quality hook and loop for secure and reliable fastening, ensuring it stays in place • The neck support cushion is made of Modal fabric and high-density sponge,
- produced using ultrasonic seamless manufacturing technology, providing softness, elasticity, and moisture-wicking properties. •The product does not contain chloroprene rubber or latex components.
- making it non-irritating to the skin. •Easy to wear, suitable for both men and women, as well as teenagers and the elderly.

www.velpeau.com

[Instructions]:

·Mild neck injury brachial neuralgia, severe neck pain

- ·Mild cervical stiffness, and chronic neck pain.
- •Correction of various pains and joint instability caused by cervical spondylosis.
- •For use during treatment or after removing a rigid neck brace.
- •Prevention of pain caused by swinging.
- Support and correction of neck posture to prevent chronic strain. •Postoperative care, torticollis, compression syndromes, rheumatic and/or degenerative diseases.

[Wear Steps]









---- www.velpeau.com -----

[Warnings]:

•Improper wearing may result in discomfort in the neck.

you experience any adverse reactions such as redness, itching, or rash, discontinue use immediately and consult a healthcare professional.

•This product is designed to be reusable, please wash the product with

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

[Washing]







---- www.velpeau.com -----







Due to the product's precision components, it is recommended not to disassemble the product during cleaning. You can disinfect or wipe the surface with alcohol or wet wines, and then allow it

•In very rare cases, this product may cause skin irritations and allergies. If

•Adjust the tightness of the brace if you experience any discomfort. water before or after each use. Store it in a well-ventilated, dry place at room temperature.

[Transportation And Storage Conditions]:













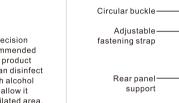

[Product Component]:



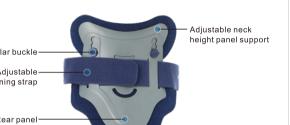

--- www.velpeau.com ----

--- www.velpeau.com -----

[Product Component]:

[Adjustment Instructions] :



Press the adjustable lock button and at the same time, use your other hand to move the lower jaw supporter up or down to achieve the appropriate support angle, then release the adjustable lock button.

---- www.velpeau.com -----

VELPEAU[®]



2 Adjustable Neck Height Panel Support Three Adjustable Settings













zhen Weprotex Technology Product Co., Ltd. : 1710, Building 11, Phase II of Tian'an Yungu

Industrial Park, Gangtou Community, Bantian Street, Longgang District, [After-sale Service]: Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen,

[Postcode]: 518129 [Tel]:+86-755-89505533 [E-mail]: sales88@velpeau.com [Website]: www.velpeau.com [Date of Preparation]: Sep.8.2023

[Modification Date]: Sep.8,2023



Regular Version

Scan the QR code
for detailed product information.



Inflatable Version

Scan the QR code
for detailed product information.