

# VELPEAU®



Hand Immobilizer Splint VP0910

**[Product Name]** : Hand Immobilizer Splint

**[Product Code]** :

- VP0910-SL       VP0910-ML       VP0910-LL
- VP0910-SR       VP0910-MR       VP0910-LR

**[Product Specifications]** :

Size	Product Code	Palm Width (inch)	Left/Right
S	VP0910-SL	2"-3"	Left
	VP0910-SR		Right
M	VP0910-ML	3"-3 1/2"	Left
	VP0910-MR		Right
L	VP0910-LL	> 3 1/2"	Left
	VP0910-LR		Right

**[Measuring Guide]** :

Please measure the widest part of your palm and refer to the provided size chart to choose the appropriate size for an optimal fit.



**[Product Compounds]** :

Polyester, Nylon, Polyurethane, Aluminum alloy.

www.velpeau.com

**[Product Features]** :

- The thumb strap, Y-shaped wrist strap, and arm strap are all filled with double-layer foam, providing high elasticity, softness, and long-lasting comfort for all-day wear, whether during the day or at night.
- Equipped with a built-in aluminum alloy resting hand splint that allows for easy adjustment of postures and angles, providing various functional positions for the fingers, hands, and wrists.
- The adjustable finger separator prevents finger curling, while the unique Y-shaped straps firmly secure the fingers, thumbs, hands, and wrists in place.
- The terry cloth lining, padded with foam, enhances grip, and prevents slippage while absorbing sweat for added comfort.

**[Application]** :


- Flaccid paralysis caused by central or peripheral neurological damage.
- Hand dysfunction after a stroke.
- Prevention or assistance in the recovery of rheumatoid deformities.
- Pain relief and inflammation reduction.
- Carpal tunnel syndrome.
- Immobilization after wrist surgery.

**[Instructions]** :

- Please choose the appropriate size, position the product on the corresponding area, and adjust the straps to ensure comfort and relieve pressure on the affected area. Then, securely fasten the straps.
- It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop fasteners.


www.velpeau.com

**[Product Component]** :

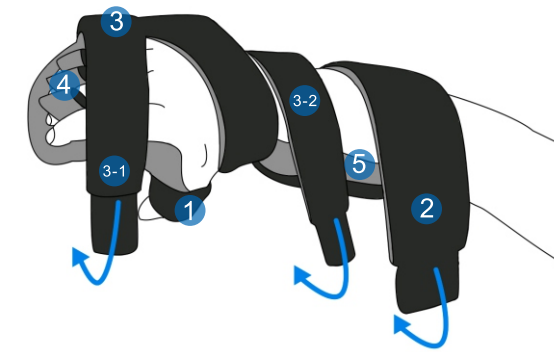

- 1 Thumb strap    4 Fingers separator    3 Y-shaped wrist strap
- 2 Arm strap (connected with hand splint)    5 Malleable aluminium hand splint
- 3-1 Wrap fingers strap    3-2 Wrap wrist strap

www.velpeau.com

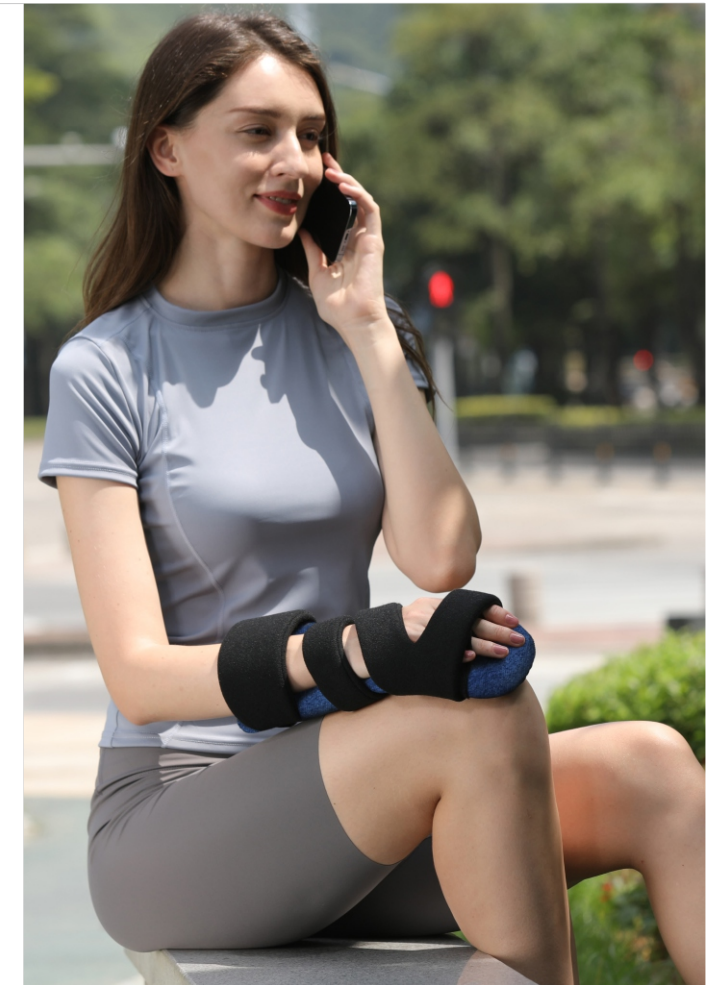

# VELPEAU®



Shenzhen Weprotex Technology Product Co., Ltd  
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755-89505533    E-mail: sales88@velpeau.com  
 www.velpeau.com    EC REP: No. 38, Boulevard Renard Benoit, 78680 Epsom, France    Made in China


**[Precautions / Warnings]** :

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

**[How to wear]** :

Before putting on the product, please note the following matters:


1. The product is equipped with a flexible aluminum support splint that can be adjusted to fit the contours of your hand. This ensures that your hand is positioned optimally for functional support.

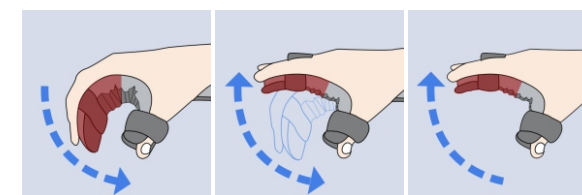

FLEXION


EXTENSION

www.velpeau.com

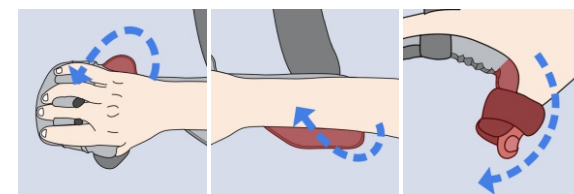

Adjust the bending of the palm side.    Adjust the bending of the arm side.    Adjust the bending of the thumb rest.

2. The finger separator can be positioned at various locations on the handrest to accommodate different hand sizes, providing a secure fit for the palm.

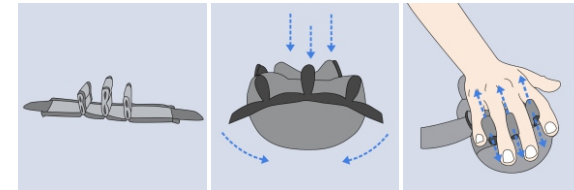

1

2

www.velpeau.com




**[Transportation And Storage Conditions]** :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

**[Washing]** :

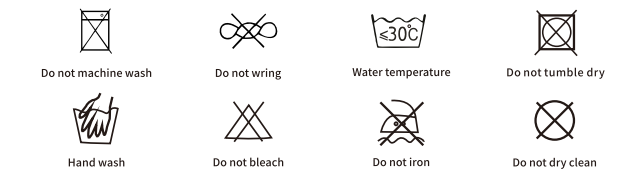

**[Term of Validity]** : 5 years

**[Production Date]** : See the package

**[Manufacturer]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[After-sale Service Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode]** : 518129

**[Tel]** : +86 - 755-89505533

**[E-mail]** : sales88@velpeau.com

**[Website]** : www.velpeau.com

**[Date of Preparation]** : Jun.9,2022

**[Modification Date]** : Jul.15,2023

www.velpeau.com