# **VELPEAU**®







□ VP0205

[Silicone Version]



□ VP0204



□ VP0204B [Stabilized Version] [Stabilized Version]



Scan the OR code to

— www.velpeau.com —

### [Product Name]: Cervical Collar ( Neck Brace ) COMFORT VERSION (VP0201)

### [Product Code]:

□VP0201-S □VP0201-M □VP0201-L □VP0201-XL

Size	Product Code	Color	Neck Circumference ( inch )	Neck Height ( inch
S	VP0201-S		10 1/2"-11 1/2"	2.75"
М	VP0201-M	Blue	11 1/2"-14"	3"
L	VP0201-L	Biue	14"-16 1/2"	3.3"
XL	VP0201-XL		16 1/2"-18 3/4"	4"

[Product Compounds]: Modal fiber, Polyurethane, Nylon.



[Product Code]:



## **ENHANCED VERSION (VP0202B)**

□ VP0202B-S □ VP0202B-M □ VP0202B-L □ VP0202B-XL

Size	Product Code	Color	Neck Circumference ( inch )	Neck Height ( inch	
S	VP0202B-S	Blue	10 1/2"-11 1/2"	2.75"	
М	VP0202B-M		11 1/2"-14"	3"	
L	VP0202B-L		14"-16 1/2"	3.3"	
XL	VP0202B-XL		16 1/2"-18 3/4"	4"	

[Product Compounds]: Modal fiber, Polyurethane, EVA foam, Nylon.





— www.velpeau.com —

### DUAL-USE VERSION (VP0203)

[Product Specifications]:

Size	Product Code	Color	Neck Circumference (inch)	Neck Height ( ir	
S	VP0203-S	Khaki -	10 1/2"-11 1/2"	2.75"	
М	VP0203-M		11 1/2"-14"	3"	
L	VP0203-L		14"-16 1/2"	3.3"	
XL	VP0203-XL		16 1/2"-18 3/4"	4"	

[Product Compounds]: Modal fiber, Polyurethane, Polypropylene, Nylon.





### SILICONE VERSION (VP0205)

[Product Code] :
□ VP0205-S □ VP0205-M □ VP0205-L

	(Prod	uct :	Speci	ficati	ons)	:
п	$\overline{}$	$\neg$			$\neg$	

Size	Product Code	Color	Neck Circumference (inch)	Neck Height ( inch
S	VP0205-S		11"-13 1/2"	2.75"
М	VP0205-M	Light Khaki	13"-16 1/4"	3"
L	VP0205-L		15 1/2"-18"	3.3"

[Product Compounds]: Silicone, Polyurethane, Modal fiber, Nylon.





www.velpeau.com —

### STABILIZED VERSION (VP0204)

[Product Code]: ☐ VP0204-S1 ☐ VP0204-M1 ☐ VP0204-L1 □ VP0204-S2 □ VP0204B-S1
□ VP0204-M2 □ VP0204B-M1
□ VP0204-L2 □ VP0204B-L1 [Product Specifications]:

Size	Product Code	Color	Neck Circumference (inch)	Neck Height ( inch )	
	VP0204-S1	Grey		3"	
s	VP0204-S2	Grey	11"-13 1/2"	,	
	VP0204B-S1	Blue	11 -10 1/2	3.5"	
	VP0204B-S2	Dide		3.3	
	VP0204-M1	Grey		3"	
	VP0204-M2	City	13 1/2"-16 1/2"	3	
М	VP0204B-M1	Blue	13 1/2 - 16 1/2	3.5"	
	VP0204B-M2	Dide		3.5	
	VP0204-L1	Grey		3"	
	VP0204-L2	Gley	40.4/0#.40.4/0#	3"	
L	VP0204B-L1	Blue	16 1/2"-19 1/2"	0.5"	
	VP0204B-L2	Dide		3.5"	

### [Product Compounds]:

Modal fiber, Polyurethane, Polyethylene, Polypropylene, Nylon



— www.velpeau.com —

# **VELPEAU**<sup>®</sup>



\*To determine the correct size, measure the circumference of your neck. Wrap the tape measure around the widest part of your neck, ensuring a snug but comfortable fit. •Measure the distance from your clavicle to your mandible to determine the

height of your neck. •Based on these measurements, refer to the size chart to select the appropriate size for the best fit.





•Various sizes are available to accommodate different neck size. \*Various sizes are available to accommodate different neck size.
 \*The V-shaped curve design follows the natural curvature of the human neck, providing a soft and resilient texture. It maintains its shape even with long-term wear, ensuring a secure and comfortable fit.
 \*The adjustable hook and loop closure allows for customized fitting and helps prevent neck stiffness. It is suitable for comfortable wear, even during sleep.

## Comfort version: Constructed with medium-density mixed foam for a soft and lightly supportive feel.

•Enhanced version:
Features a dual-layer design with medium-density mixed foam and soft, high-elasticity EVA foam for increased support and comfort.

\*Dual-use version: Includes a detachable plastic support strap that adds additional support when attached and can be used as a Comfort version when removed.

•Silicone version:
Made from skin-friendly fibers and featuring a soft and flexible latex-free sponge material, this product is designed to prevent skin irritation and provide excellent ventilation. It keeps the neck dry and cool, making it perfect for extended use.

— www.velpeau.com —

### [Application]:

•Mild neck injuries, neck brachial neuralgia, and neck pain.

Mild cervical arthritis and chronic neck pain.

Various types of pain associated with cervical spondylosis and joint

instability.
•Post-treatment phase following the removal of a rigid neck brace. Prevention of pain caused by excessive movement or swinging.
 Support and correction of neck posture to prevent chronic strain.

•Postoperative care, torticollis, compression syndromes, and rheumatic and/or degenerative disorders.

•Improper wearing may result in discomfort in the neck.

In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue use immediately and consult a healthcare professional. •Adjust the tightness of the brace if you experience any discomfort. •This product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.

A detachable and washable sleeve is included with the product, providing added comfort and hygiene.

•Position the V-groove of the product under your chin and secure it in place

using the hook and loop strap. •Adjust the hook and loop strap to achieve the desired level of tightness and ensure maximum comfort for your neck.

- www.velpeau.com ----










— www.velpeau.com ——

Silicone	VP0205	Khaki	4 <b>44</b> 4	****	*****	>	>	>	<b>&gt;</b>	>
Stabilized	VP0204	Grey/Blue	****	****	****	>	>	>	<b>&gt;</b>	>
Dual-use	VP0203	Khaki	*****	****	****	>	>	>	>	>
Enhanced	VP0202B	Blue	****	****	****		>	>	>	>
Comfort	VP0201	Blue	****	****	****	>	>	>	>	
Version	Product No.	Colors	Support	Comfort	Breathability	Sleep	Office	Travel	Household	Medical

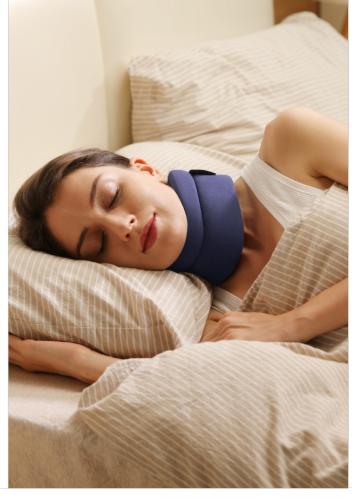

### 【 Washing 】





X





<30°C







[Term of Validity]: 5 years

[Production Date]: See the package [Manufacturer]: Shenzhen Weprotex Technology Product Co., Ltd. [Production Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District,

[After-sale Service]: Shenzhen Weprotex Technology Product Co., Ltd. [After-sale Service Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen,

[Postcode]: 518129 [Tel]: +86 - 755-89505533 [E-mail]: sales88@velpeau.com [Website]: www.velpeau.com [Date of Preparation]: Sep.15.2022 [Modification Date]: Jul.15,2023

Shenzhen, China

www.velpeau.com ---