

VELPEAU®



Neck Brace
VP0205

[Product Name] : Cervical Collar - Silicone Version (Neck Brace)

[Product Code] :


VP0205-S VP0205-M VP0205-L

[Product Specifications] :

Size	Product Code	Color	Neck Circumference (inch)
S	VP0205-S	Light Khaki	11"-13 1/2"
M	VP0205-M		13"-16 1/4"
L	VP0205-L		15 1/2"-18"

How to choose the right size for you?

Measure the neck circumference with a soft ruler. And the neck height is the distance from the collarbone to the mandible. Finally choose the right size according to the measurement results.

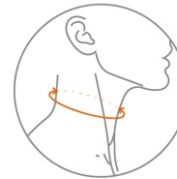

Neck Height

[Fiber Content] : Silicone, Polyurethane, Polyester, Spandex, Nylon.

[Product Performance] :

- Multiple sizes available for different neck circumference and neck height.
- V-shaped curve design according to the curvature of the human neck, the texture is very soft and resilient. It is not easily deformed with long-term wearing and it fits firmly.
- Adjustable hook and loop closure to prevent neck stiffness, which is also comfortable at night.
- Made of medical silicone with a very soft and flexible texture. Free from neoprene and latex, it is non-irritating to the skin. Good ventilation, keep your neck dry and cool, more suitable for use in the hot summer.

www.velpeau.com

[Indications] :

- Benign neck injury, neck brachial neuralgia and neck pain.
- Mild cervical arthrosis, chronic neck pain.
- All kinds of pain caused by cervical spondylosis and the regulation of joint instability.
- Treatment phase after lifting the rigid neck brace.
- To prevent the pain caused by swinging.
- Support and correct neck posture, prevent chronic strain.
- Postoperative care, torticollis, compression syndromes, rheumatic and/or degenerative disorders.

[Wear Steps] :

- Please place the V-groove under your chin and stick to back Velcro.
- Check if it's too tight or too loose, and adjust the Velcro to ensure comfortable and stable wearing.



[Precautions / Warnings] :

- Incorrect installation may cause it to be too tight.
- Skin irritations and allergies can occur in very rare cases.
- If discomfort occurs in using process, please adjust the tightness.
- Wash it with water before or after using, store it in a ventilated dry place at normal temperature, and it is reusable.

www.velpeau.com

VELPEAU®




Shenzhen Weprotex Technology Product Co., Ltd
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China
 Tel: 0755 - 89505533 E-mail: sales88@velpeau.com
 www.velpeau.com
 EC REP: No. 38, Boulevard Renard Benoit, 78880 Epone, France Made in China



[Notes] :

A separately washable removable sleeve is provided to be positioned on the product, which is more comfortable and hygienic.

[Washing] :

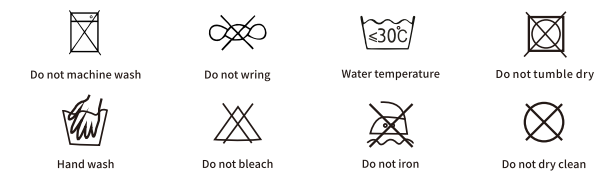

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[After Sale Service Unit] : Shenzhen Weprotex Technology Product Co., Ltd.

[After Sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Sep.15,2022

[Modification Date] : Mar.29,2023

www.velpeau.com