

# VELPEAU®



Arm Slings  
VP0301



**[Product Name]** : Arm Slings (Shoulder Immobilizer)

**[Product Code]** :


VP0301-S     VP0301-M     VP0301-L     VP0301-XL

**[Product Specifications]** :

Size	Product Code	Bust(inch)	Left/Right
S	VP0301-S	20"-27 1/2"	Fits both
M	VP0301-M	27 1/2"-39 1/2"	
L	VP0301-L	39 1/2"-51 1/4"	
XL	VP0301-XL	>51 1/4"	

**How to choose the suitable one?**

Please choose the right size according to your bust. If the product is worn outside of clothes, please measure the bust circumference after putting on the clothes.

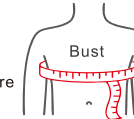

**[Product Features]** :

- Provides stability and fixation for the shoulder and upper arm.
- Features chest straps for enhanced stability.
- Available in multiple sizes to accommodate different body types.
- Filled with sponges for a lightweight and comfortable experience, ideal for sleep.
- The back cross design prevents slipping.
- Thumb hook design for balanced and supported arm positioning.
- Includes a practical small storage bag.
- Equipped with a high-viscosity hook and loop fastener for easy wearing and adjustment.
- Suitable for both left and right arms.

www.velpeau.com

# VELPEAU®



**Shenzhen Weprotex Technology Product Co.,Ltd**  
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755 - 89505533    E-mail: sales88@velpeau.com  
 www.velpeau.com  
 EC REP: No. 38, Boulevard Renard Benoît, 78680 Epone, France    Made in China

**[Product Compounds]** :

Polyester, Polyurethane, Cotton, Nylon.

**[Applications]** :

- Shoulder contusions and sprains.
- Dislocation of shoulder joint.
- Rotator cuff injury.
- Humeral caput fracture.
- Shoulder blade fracture.
- Postoperative and plaster fixation.
- Support for nerve paralysis or arm hemiplegia.

**[How to wear]** :



⚠ When wearing, please note that the side with the pocket is facing your body.

www.velpeau.com



**[Precautions / Warnings]** :

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

**[Instructions]** :


- When wearing, ensure that the side with the pocket is facing your body.
- Loosen the belt before placing the arm in the corresponding position then adjust the shoulder straps for comfort and tension-free fit, then secure the straps.
- Check the strap to ensure it is neither too tight nor too loose, as excessive tightness can impede blood circulation and hinder wound recovery, while looseness may compromise stability.
- Adjust the height of the shoulder straps as advised by the doctor, as a 90-degree suspension helps improve blood circulation and reduce swelling.
- The chest and arm fixed belt can be removed as needed for added flexibility.
- Assembling the product is simple, requiring only fastening the hook and loop strap.

www.velpeau.com

**[Transportation and Storage Conditions]** :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

**[Washing]** :

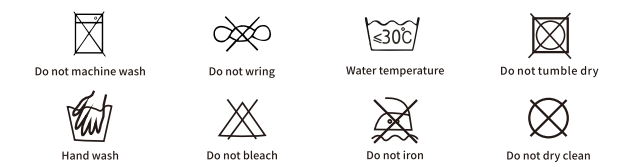

**[Term of Validity]** : 5 years

**[Production Date]** : See the package

**[Manufacturer]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[After-sale Service Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode]** : 518129

**[Tel]** : +86 - 755-89505533

**[E-mail]** : sales88@velpeau.com

**[Website]** : www.velpeau.com

**[Date of Preparation]** : Jan.9,2017

**[Modification Date]** : Jul.15,2023

www.velpeau.com