

VELPEAU®



Arm Sling
VP0302



[Product Name] : Arm Slings

[Product Code] :

VP0302-S VP0302-M VP0302-L

[Product Specifications] :

Size	Product code	Shoulder width (inch)	Left/Right
S	VP0302-S	10 1/2"-12 1/2"	Reversible
M	VP0302-M	12 1/2"-17 3/4"	
L	VP0302-L	17 3/4"-21 3/4"	

[Product Performance] :

- Made of breathable, durable and comfortable fabric;
- Hang the upper limb with the arm sling, make it fixed;
- Simple wearing and fit well;
- Tension adjustable, applied with either hand, suitable for most patients.
- Built-in thumb loops support your hand to prevent fatigue.
- Can be worn on the left or right arms.

[Fiber Content] :

Polyester, Cotton, Nylon, Polypropylene, Polyurethane.

www.velpeau.com

VELPEAU®



Shenzhen Weprotex Technology Product Co., Ltd

Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

Tel: 0755-89505533 E-mail: sales88@velpeau.com

www.velpeau.com

EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France Made in China

[Instructions] :

- Please use it under the guidance of doctors and professional persons.
- Select the appropriate size, loosen the belt after the arm is on the corresponding position, and adjust the shoulder straps to ensure the affected part is comfortable and without tensions and then paste the strap.
- Check the strap in case it will be too tight or too loose, to ensure the stability after wearing.
- Adjust the height of the shoulder straps according to the doctor's advice. (90 degrees suspension is conducive to blood circulation and helpful to reduce swelling)
- Simple assembling and just need to fasten the Velcro.

[Precautions / Warnings] :

- To be used by adults and adolescents.
- Easy to use, it can be used with the help of medical personnel or healthy people when you have difficulty on your own.
- Place the fixed part as priority. If discomfort occurs in using process, please adjust the tightening.
- Wash it with water before or after using, store it in a ventilated dry place at normal temperature, and it's reusable.
- This product is for isolated use, with no other combination of product using, and its safety and effectiveness will not be influenced by other external conditions while using.
- No specific special skills, knowledge or training required for users, it does not contain materials with allergy or side effects to human body, which is in accordance with ROHS standard.
- If there is an open wound on the skin, use the product carefully or bandage the wound to avoid direct contact with it.

www.velpeau.com



[Indications] :

- The rehabilitation of a dislocated shoulder, scapula fracture after surgery.
- The fixation of nerve paralysis and elbow injury.
- The fixation of forearm fractures, fractures of the humerus, fracture of the wrist and fingers after surgery and the use during gypsum period.

[Transportation and Storage Conditions] :


It is allowed to be transported by general means of transportation and stored at normal temperature, but it must be fireproof, moisture-proof, dustproof, rodent proof, and insect proof.

[Wear steps] :



www.velpeau.com

[Washing] :

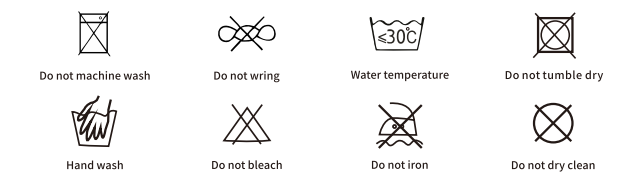

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[After Sale Service Unit] : Shenzhen Weprotex Technology Product Co., Ltd.

[After Sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Jan.9.2017

[Modification Date] : Mar.22,2023

www.velpeau.com