VELPEAU®



[Product Name] : Clavicle Brace (Posture Corrector Brace)

[Product Code]:

□ VP0303-S □ VP0303-M □ VP0303-L □ VP0303-XL

[Product Specifications]:

Size	Product code	Color	Shoulder width (inch)
S	VP0303-S	White	10"-13"
М	VP0303-M		13"-15 1/2"
L	VP0303-L		15 1/2"-19"
XL	VP0303-XL		19"-23"

[Product Features]

- •This product utilizes the Figure 8 bandage design principle for clavicle and shoulder recovery after injury or surgery.
- •It features three T-shaped buckles and hook & loop (Velcro) fastens for easy adjustment and customization of the clavicle strap's direction
- •The shoulder straps and back pads are equipped with sponge straps for skin-friendly, breathable, and sweat-absorbent comfort.
- •The straps help align the fractured bones by correctly promoting proper shoulder alignment.

www.velpeau.com

[Applications]:

- For clavicle fracture, reduction and fixation of shoulder injuries or after surgery.
- · Acromioclavicular dislocation.
- •Correct shoulder posture to prevent hunchback.

[Product Compounds]:

Polyurethane, Sponge, Felt fabric, Polyamide, Hook and loop fastener,

[Instructions] :


- •Begin by threading the two shoulder straps through the clasps and securing them on the outside of the straps.
- ·Slide one arm through the loop and then repeat with the other arm,
- positioning the product on the shoulders.
- •Use the hook and loop on the shoulder strap to adjust the tightness as needed, ensuring a comfortable and tension-free fit.
- •For a step-by-step guide on how to wear the product, visit our official website at www.velpeau.com and watch the comprehensive

VELPEAU[®]



Shenzhen Weprotex Technology Product Co.,Ltd

www.velpeau.com

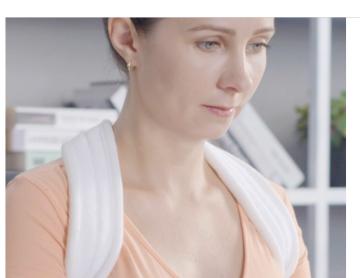

[Front View]:



[Back View] :



www.velpeau.com

[Precautions / Warnings]:

- $\bullet \hbox{Please consult a professional for guidance before using this product}. \\$
- This product is intended for adult and adolescent use
- •It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- •If any discomfort occurs during use, adjust the tightness accordingly.
- •The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- •This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- $\bullet \hbox{No specific special skills, knowledge, or training required for users. The }$ product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- •If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

[Transportation and Storage Conditions]:

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com -

【Washing】:

Hand wash





X











[Term of Validity]: 5 years

[Production Date]: See the package

[Manufacturer]: Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service]: Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen,

[Postcode]: 518129

[Tel]: +86 - 755-89505533

[E-mail]: sales88@velpeau.com [Website]: www.velpeau.com

[Date of Preparation]: Jam.9,2017

[Modification Date]: Jul.15,2023

www.velpeau.com -