VELPEAU®



[Product Name] : Shoulder Sling (Shoulder Support Brace)

[Product Code]:

□ VP0304-S □ VP0304-M □ VP0304-L □ VP0304-XL

[Product Specifications]:

Size	Product code	Bust (inch)	Left/Right
S	VP0304-S	23 1/2"-29 1/2"	Fits both
М	VP0304-M	27 1/2"-35 1/2"	
L	VP0304-L	35 1/2"-47 1/4"	
XL	VP0304-XL	>47 1/4"	

[Product Features]:

- •This product effectively immobilizes the shoulder joint.
- •It features a high-strength and skin-friendly hook and loop closure, allowing for easy on and off and adjustable tightness for a customized fit.
- •The entire product is filled with soft sponge material, ensuring comfort
- and is also suitable for sleep.
- ·Includes a convenient small storage bag.
- •Designed with perforated fabric for enhanced ventilation.
- •Neoprene-free and latex-free, preventing skin irritation.
- ·Can be worn on either the left or right arm.

[Indications]:

- •Shoulder joint trauma, rheumatoid diseases and postoperation. •Injury and postoperation of rotator cuff, acromioclavicular joint or clavicle.
- •Support for nerve paralysis or arm hemiplegia.

[Applications]:

- •Proper sizing can be achieved by following the wear step diagram in the manual or watching the detailed wearing video on the official website www.velpeau.com.
- It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- •The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop fasteners.

[Product Compounds]:

Polyester, Cotton, Nylon, Polyurethane.

VELPEAU®



Shenzhen Weprotex Technology Product Co.,Ltd



www.velpeau.com

www.velpeau.com

[Wear Steps]









www.velpeau.com -

[Precautions / Warnings] :

- $\bullet \mbox{Please consult a professional for guidance before using this product. } \\$
- This product is intended for adult and adolescent use only.
- •It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- •The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- •This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- •No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- ${}^{\textstyle \bullet}$ If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

[Transportation and Storage Conditions]:

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com -

【Washing】:













Do not dry clean

[Term of Validity]: 5 years

[Production Date]: See the package

[Manufacturer]: Shenzhen Weprotex Technology Product Co., Ltd. [Production Address]: 1710, Building 11, Phase II of Tian'an Yungu

Industrial Park, Gangtou Community, Bantian Street, Longgang District,

[After-sale Service]: Shenzhen Weprotex Technology Product Co., Ltd. [After-sale Service Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen,

[Postcode]: 518129 [Tel]: +86 - 755-89505533 [E-mail]: sales88@velpeau.com [Website]: www.velpeau.com [Date of Preparation]: Jam.9,2017 [Modification Date]: Jul.8,2023

www.velpeau.com