





## [Product Specifications] Size Product Code Left/Right Underbust(inch) VP0309-U1-S-L Left VP0309-U1-S-R Right 24"-29.5" Right VP0309B-S-R VP0309-U1-M-L Left VP0309-U1-M-R Right VP0309B-M-L Left 29.5"-41" VP0309B-M-R VP0309-U1-L-L Left VP0309-U1-L-R Right VP0309B-L-L Left VP0309B-L-R Right 41"-53"





# •2XS/XS sizes are suitable for kids. Please choose the size based on your arm length for the optimal fit. -S/M/L sizes are suitable for adults. Please choose the size based on your underbust size for the optimal fit. [Product Compounds]

Polyamide, Modal fiber, Polyurethane, Nylon, Cotton, Polyester, Polypropylene.

Polyester, Nylon, Cotton, Polypropylene.

## [Product Features]

- \*Utilizes a T-shaped shoulder and back strapping system that brings the forearm closer to the body, reducing pressure on the neck and shoulders.
- and breathability. •Quick and convenient arm suspension angle adjustment. •Equipped with two hand grip straps in the arm pocket for improved arm stability.
  •Built-in thumb loops provide support to prevent hand fatigue.

### [Indications]

Acute shoulder instability.

•Muscle and/or tendon pain injury, shoulder disorders. •Immobilisation of the Shoulder & Elbow.

•neoprene and latex free to prevent skin irritation.

•Postoperative care of rotator cuff surgery. •Distortion, contusion, luxation of the shoulder.

•Trauma to the Shoulder & Elbow.

- •Proper sizing can be achieved by following the wear step diagram in the manual or watching the detailed wearing video on the official website
- It is important to check the strap tightness to ensure stability. Avoid making
  it too tight which can restrict blood flow, hinder wound healing, or loose which
- ${}^{\bullet} \text{The product is designed for easy assembly and disassembly requiring simple} \\$ attachment or detachment of the hook and loop fasteners.

## [Precautions / Warnings]:

- Please consult a professional for guidance before using this product.

  This product is intended for adult and adolescent use only.
- medical personnel or others when needed. •If any discomfort occurs during use, adjust the tightness accordingly.
  •The product is designed to be reusable, please wash the product with
- water before or after each use. Store it in a well-ventilated, dry place at •This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external
- •No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which
- is free from allergens or side effects on the human body. •If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

### [Transportation and Storage Conditions]:

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

## 【 Washing 】









## [Production Date ]: See the package

[Manufacturer]: Shenzhen Weprotex Technology Product Co., Ltd. [Production Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service]: Shenzhen Weprotex Technology Product Co., Ltd. [After-sale Service Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode]: 518129 [Tel]: +86 - 755-89505533

[E-mail]: sales88@velpeau.com [Website]: www.velpeau.com [Date of Preparation]: Mar.21,2023

[Modification Date]: Mar.21,2023