

# VELPEAU®



Wrist Brace  
VP0901



**[Product Name]** : Wrist Brace( Wrist Immobilizer )

**[Product Code]** :

- VP0901-SL     VP0901-ML     VP0901-LL  
 VP0901-SR     VP0901-MR     VP0901-LR

**[Product Specifications]** :

Product No	Product code	Size	Wrist circumference (inch)	Left / Right
VP0901	VP0901-SL	S	4 3/4"- 6 1/4" / 12-16	Left
	VP0901-SR			Right
	VP0901-ML	M	6 1/4"- 8 1/2" / 16-22	Left
	VP0901-MR			Right
	VP0901-LL	L	>8 1/2" / >22	Left
	VP0901-LR			Right

**[Product Performance]**

- Open sleeve design makes the wearing more easy and convenient, light and breathable and comfortable.
- The wrist is fixed and stabilized by plastic aluminum alloy splint.
- Hook and loop fastener design, adjustable tightness, excellent material.

**[Fiber Content]** :


Polyester, Polyurethane, Cotton, Nylon, Polypropylene, Aluminum alloy.

# VELPEAU®



**Shenzhen Weprotex Technology Product Co.,Ltd**

Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755 - 89505533    E-mail: sales88@velpeau.com  
 www.velpeau.com  
 EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France    Made in China

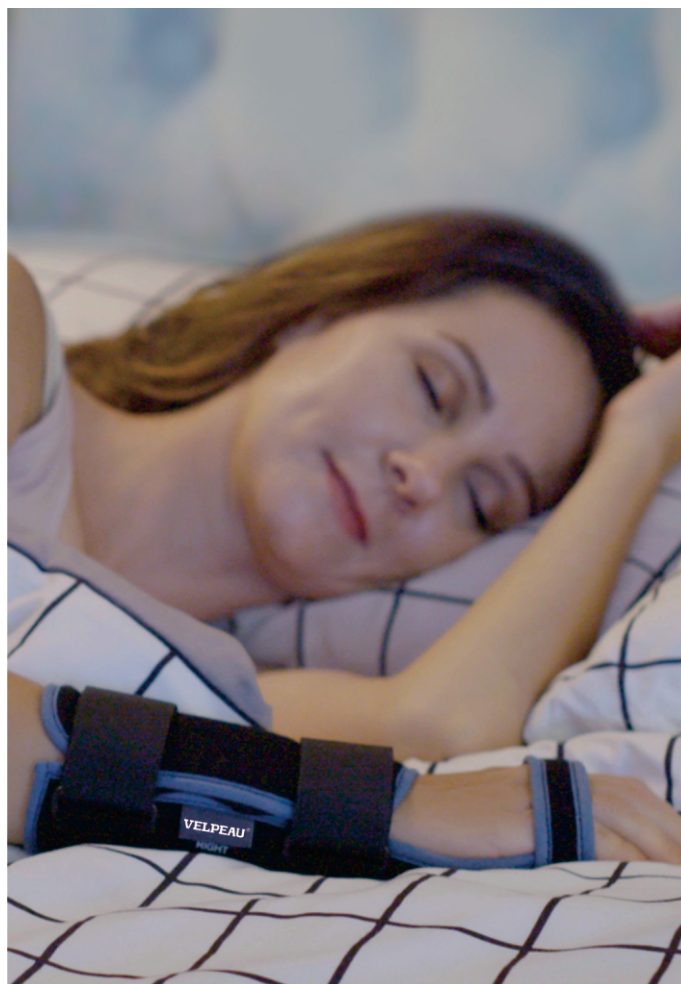

**[Indications]** :

- Wrist fracture, radial/ulna distal fracture.
- Wrist soft tissue injury.
- Carpal tunnel syndrome.
- Arthritis of the wrist, tendonitis and tenosynovitis.
- paralysis of radial nerve
- Use after removal of the cast or splint.

**[Instructions]** :

- Please use it under the guidance of doctors and professional persons.
- Select the appropriate size, put the product on the corresponding position, and adjust the straps to ensure the affected part is comfortable with no pressure and then attach the strap.
- Check the strap in case it will be too tight or too loose, to ensure the stability after wearing. If it is too tight, it will affect blood flow, which is not conducive to wound healing, or if it is too loose, it can't be fixed.
- Easy to assemble and disassemble, just put on or take off the Velcro.

**[Wear Steps]** :




**[Precautions / Warnings]**

- To be used by adults and adolescents.
- Easy to use, it can be used with the help of medical personnel or healthy people when you have difficulty on your own.
- Place the fixed part as priority. If discomfort occurs in using process, please adjust the tightening.
- Wash it with water before or after using, store it in a ventilated dry place at normal temperature, and it's reusable.
- This product is for isolated use, with no other combination of product using, and its safety and effectiveness will not be influenced by other external conditions while using.
- No specific special skills, knowledge or training required for users, it does not contain materials with allergy or side effects to human body, which is in accordance with ROHS standard.
- If there is an open wound on the skin, use the product carefully or bandage the wound to avoid direct contact with it.

**[Transportation And Storage Conditions]**

It is allowed to be transported by general means of transportation and stored at normal temperature, but it must be fireproof, moisture-proof, dustproof, rodent proof, and insect proof.

**[Washing]** :

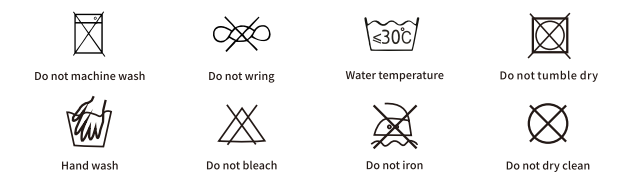

**[Term of Validity]** : 5 years

**[Production Date]** : See the package

**[Manufacturer]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

**[After Sale Service Unit]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[After Sale Service Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode]** : 518129

**[Tel]** : +86 - 755-89505533

**[E-mail]** : sales88@velpeau.com

**[Website]** : www.velpeau.cn

**[Date of Preparation]** : Jan.9,2017

**[Modification Date]** : Mar.14,2023