

# VELPEAU®



Thumb Wrist Brace VP0902B



**[Product Name] :**

Thumb Wrist Brace ( Short Version )

**[Product Code] :**

- VP0902B-SL    VP0902B-ML    VP0902B-LL
- VP0902B-SR    VP0902B-MR    VP0902B-LR

**[Product Specifications] :**

Size	Product code	Around Wrist(inch)	Left/Right
S	VP0902B-SL	5"- 6"	Left
	VP0902B-SR		Right
M	VP0902B-ML	6"- 8 1/2"	Left
	VP0902B-MR		Right
L	VP0902B-LL	> 8 1/2"	Left
	VP0902B-LR		Right

**[Product Features] :**

- Anatomically shaped to provide functional support for the wrist while allowing free movement of the fingers.
- Two aluminum alloy splints on the palm, along with elastic support strips on the back of the hand, provide effective control of palmar and dorsal flexion.
- Independent and removable thumb splint with adjustable thumb straps to maintain the thumb in a stable position and limit its movement, ensuring moderate flexibility while stabilizing the joint.
- The two thumb straps can be unwrapped as needed to release the thumb without removing the entire brace, making it easy to manage.
- The entire brace is padded with sponges for optimal comfort, and it is made without neoprene and latex which minimizing the risk of skin irritation.

www.velpeau.com

# VELPEAU®



**Shenzhen Weprotex Technology Product Co.,Ltd**  
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755 - 89505533   E-mail: sales88@velpeau.com  
 www.velpeau.com  
 EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France   Made in China

**[Product Comounds] :**

Polyester, Polyurethane, Cotton, Nylon, Polypropylene, Aluminum alloy, Polycarbonate.

**[Application] :**

The wrist brace with thumb-spica splint can be used for all indications that require immobilisation of the wrist with thumb fixation, such as: •Sprain, distorsion or stable fractures of the wrist and / or thumb.

- Post-operative immobilization of the wrist and thumb.
- De Quervain's Tenosynovitis.
- Carpal tunnel syndrome (CTS).
- Lesions of the collateral ligaments of the basal thumb joint.
- Tendinitis and tenosynovitis of the wrist and / or thumb.
- Radial nerve palsy of hand.
- Inflammatory phase of rheumatic diseases.
- After postoperative cast or splint removal.

**[Precautions / Warnings] :**

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

www.velpeau.com



**[How to wear] :**



**[Instructions] :**


- Proper sizing can be achieved by following the wear step diagram in the manual or watching the detailed wearing video on the official website [www.velpeau.com](http://www.velpeau.com).
- It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop fastener.

www.velpeau.com

**[Transportation And Storage Conditions] :**

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

**[Washing] :**

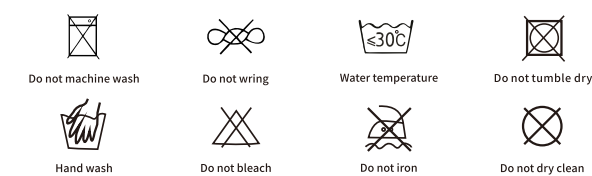

**[Term of Validity] :** 5 years

**[Production Date] :** See the package

**[Manufacturer] :** Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service] :** Shenzhen Weprotex Technology Product Co., Ltd.

**[After-sale Service Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode] :** 518129

**[Tel] :** +86 - 755-89505533

**[E-mail] :** sales88@velpeau.com

**[Website] :** [www.velpeau.com](http://www.velpeau.com)

**[Date of Preparation] :** Sep.15,2022

**[Modification Date] :** Jul.15,2023

www.velpeau.com