

# VELPEAU®



Wrist Brace  
VP0903



**[Product Name]** : Wrist Brace ( Double Drawstring Version)

**[Product Code]** :

- VP0903-SL       VP0903-ML       VP0903-LL  
 VP0903-SR       VP0903-MR       VP0903-LR

**[Product Specifications]** :

Size	Product Code	Around Wrist(inch)	Left/Right
S	VP0903-SL	4 3/4"- 6 1/4"	Left
	VP0903-SR		Right
M	VP0903-ML	6 1/4"-8 1/2"	Left
	VP0903-MR		Right
L	VP0903-LL	> 8 1/2"	Left
	VP0903-LR		Right

**[Product Features]** :

- Anatomically shaped to provide functional support for the wrist while allowing free movement of the fingers.
- The functional drawstring closure method enhances the breathability and lightweight nature of the product. It allows for easy one-handed wearing and removal while providing the ability to quickly adjust the pressure on the wrist.
- Two aluminum alloy splints on the palm, along with elastic support strips on the back of the hand, provide effective control of palmar and dorsal flexion.
- The entire brace is padded with sponges for optimal comfort, and it is made without neoprene and latex which minimizes the risk of skin irritation.

www.velpeau.com

# VELPEAU®



**Shenzhen Weprotex Technology Product Co., Ltd**  
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755 - 89505533      E-mail: sales88@velpeau.com  
 www.velpeau.com  
 EC REP: No. 38, Boulevard Renard Benoit, 76680 Epone, France      Made in China

**[Product Componds]** :

**[Main material]** : Polyamide, Modal fiber, Polyurethane  
**[Accessories materials]** : Nylon, Cotton, Spandex, Polyamide, PP, Aluminum alloy, PE, Polyester, Rubber band, PVC.etc

**[Application]** :

- Wrist fractures, including radius/ulna distal fractures.
- Wrist soft tissue injuries.
- Carpal tunnel syndrome.
- Wrist arthritis, tendonitis, and tenosynovitis.
- Radial nerve parlay.
- Post-removal of casts or splints.

**[Warning]** :

- Do not use this product on the skin that has skin diseases, open wounds, or injuries in the application area.
- This product is intended for self-use only and should not be shared.
- Consult with a healthcare professional before using this product in the following cases:
  1. Blood circulation and sensory disorders in the wrist area, such as diabetes.
  2. Arterial vascular diseases (Arterial occlusive disease).
  3. Venous vascular diseases, such as phlebitis and venous thrombosis.
  4. Disorders of the lymphatic system in the arms.
  5. Allergic reactions to any components of the product.
  6. Specific morphological peculiarities in the wrist being treated may restrict or complicate treatment with the support and its proper positioning.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue immediately and consult a healthcare professional.

**[Important Factors]** :

- If experience any discomfort during the use of the product, please adjust the tightness according to optimal comfort and support.
- This product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.

www.velpeau.com



**[How to wear]** :




**[Transportation And Storage Conditions]** :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

**[Washing]** :

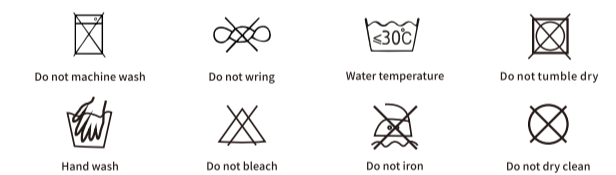

**[Term of Validity]** : 5 years

**[Production Date]** : See the package

**[Manufacturer]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service]** : Shenzhen Weprotex Technology Product Co., Ltd.

**[After-sale Service Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode]** : 518129

**[Tel]** : +86 - 755 - 89505533

**[E-mail]** : sales88@velpeau.com

**[Website]** : www.velpeau.com

**[Date of Preparation]** : May.25,2023

**[Modification Date]** : Jul.15,2023

www.velpeau.com