

VELPEAU®



[Product Name] : Thumb Stabilizer

[Product Code] :

VP0905-S VP0905-M VP0905-L

[Product Features] :

Size	Product Code	Around Wrist(Inch)	Left/Right
S	VP0905-S	5"- 6"	Fits both
M	VP0905-M	6"- 8 1/2"	
L	VP0905-L	> 8 1/2"	

[Product Performance] :

- The product is built with a high elastic nylon plastic plate designed to support and stabilize the thumb's joint between the thumb metacarpal bone and the thumb proximal phalanx.
- It features a comfortable thumb loop and strap that can be adjusted for personalized support.
- The materials used are soft, breathable, and durable, ensuring long-lasting comfort.
- The low-profile design allows for unrestricted movement of the remaining fingers.
- Putting on and taking off the product is easy and convenient.
- It is suitable for use on both the left and right hands.

[Product Compounds] :

Polyester, Polyamide, Cotton, Polyurethane, Polypropylene, Nylon.

www.velpeau.com

VELPEAU®



Shenzhen Weprotex Technology Product Co., Ltd
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China
 Tel: 0755-89505533 E-mail: sales88@velpeau.com
 www.velpeau.com
 EC REP: No. 38, Boulevard Renard Benoit, 78660 Epone, France Made in China

[Applications] :

- This thumb stabilizer provides comfortable protection and moderate support for weak, aching or sore thumbs.
- For use on soft tissue injuries, ligament strains, gamekeepers thumb, osteoarthritis and degenerative joint disease.
- It is also ideal for people with arthritic symptoms or pain associated with repetitive stress such as texting and gaming.
- Provides warmth to the thumb area and joint.

[Instructions] :

- Please choose the appropriate size, position the product on the corresponding area, and adjust the straps to ensure comfort and relieve pressure on the affected area. Then, securely fasten the straps.
- It is important to check the strap tightness to ensure stability. Avoid tightness, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop (Velcro) fasteners.

[Precautions / Warnings] :

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

www.velpeau.com



[Wear Steps] :




www.velpeau.com

[Transportation And Storage Conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

[Washing] :

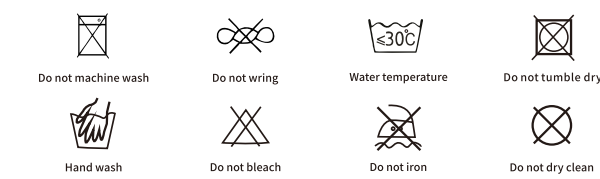

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Sep.15.2022

[Modification Date] : Jul.8.2023

www.velpeau.com