

VELPEAU®



[Product Name] : Thumb Wrist Brace(Breathable Version)

[Product Code] :

VP0906-SL VP0906-ML VP0906-LL
 VP0906-SR VP0906-MR VP0906-LR

[Product Specifications] :

Size	Product Code	Around Wrist(inch)	Left/Right
S	VP0906-SL	<6 3/4"	Left
	VP0906-SR		Right
M	VP0906-ML	6 3/4"-9 1/2"	Left
	VP0906-MR		Right
L	VP0906-LL	> 9 1/2"	Left
	VP0906-LR		Right

[Product Features] :

- The unique Y-shaped straps technique enhances the ease and convenience of adjusting and fitting the product.
- Two V-shaped soft support strips integrated into the thumb area provide lightweight, thin, and flexible support. They promote joint flexibility while ensuring effective stabilization.
- The entire product is constructed with breathable 3D mesh fabric, offering a lightweight and breathable experience, preventing discomfort from excessive heat. This makes it well-suited for prolonged use during the summer or while sleeping.
- The removable palm splint allows for versatility and customization.
- The product features a lightweight design that is space-saving and easy to clean.

www.velpeau.com

VELPEAU®



Shenzhen Weprotex Technology Product Co.,Ltd
Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China
Tel: 0755-89505533 E-mail: sales88@velpeau.com
www.velpeau.com
EC REP: No. 38, Boulevard Renard Benoit, 78660 Epone, France Made in China

[Product Compounds] :

Polyester, Cotton, Nylon, Polypropylene, Polyurethane, Aluminum alloy, Carbon fibre.


[Application] :

- Tendonitis in the hand region, tenosynovitis.
- Post-traumatic treatment of moderate to severe sprains.
- At the beginning of carpal tunnel syndrome(CTS).
- Activated osteoarthritis or arthritis of the wrist.
- Instability and functional overloading in the wrist area.
- Radial nerve palsy of hand.
- Follow-up treatment for wrist fractures, after surgery or after wearing a plaster cast.
- Stage I complex regional pain syndrome (CRPS).

[Warning] :

- Do not use this product on the skin that has skin diseases, open wounds, or injuries in the application area.
- This product is intended for self-use only and should not be shared.
- Consult with a healthcare professional before using this product in the following cases:
 1. Blood circulation and sensory disorders in the wrist area, such as diabetes.
 2. Arterial vascular diseases (Arterial occlusive disease).
 3. Venous vascular diseases, such as phlebitis and venous thrombosis.
 4. Disorders of the lymphatic system in the arms.
 5. Allergic reactions to any components of the product.
 6. Specific morphological peculiarities in the wrist being treated may restrict or complicate treatment with the support and its proper positioning.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue immediately and consult a healthcare professional.

www.velpeau.com

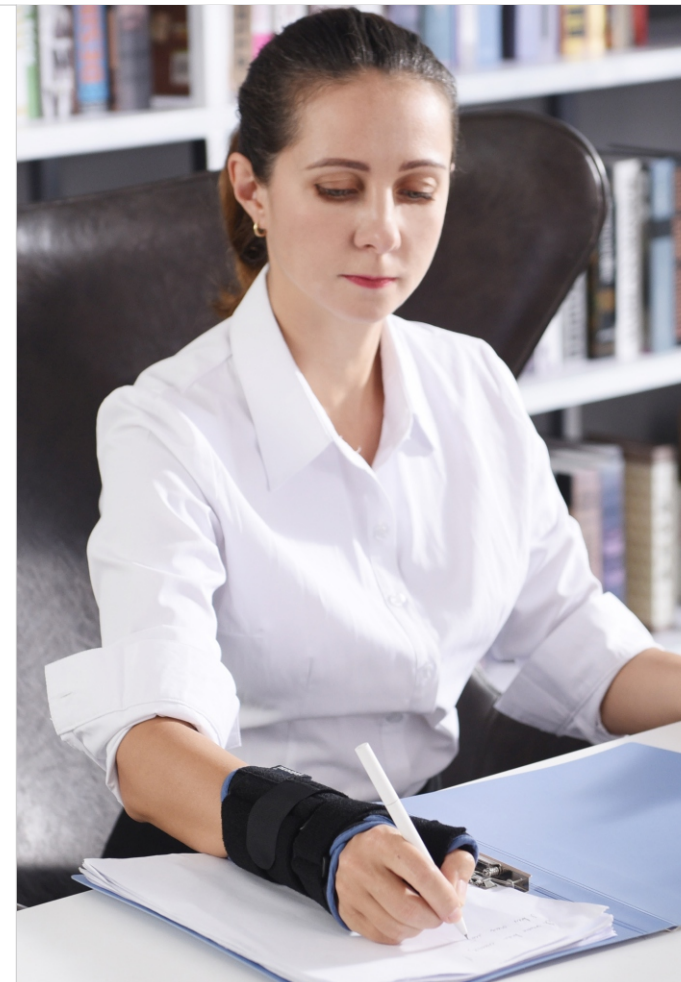

[Important Factors] :

- If experience any discomfort during the use of the product, please adjust the tightness according to optimal comfort and support.
- This product is designed to be reusable, please wash the product with* water before or after each use. Store it in a well-ventilated, dry place at room temperature.

[How to wear] :



[Instructions] :


- Please choose the appropriate size, position the product on the corresponding area, and adjust the straps to ensure comfort and relieve pressure on the affected area. Then, securely fasten the straps.
- It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop fasteners.

www.velpeau.com

[Transportation And Storage Conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

[Washing] :

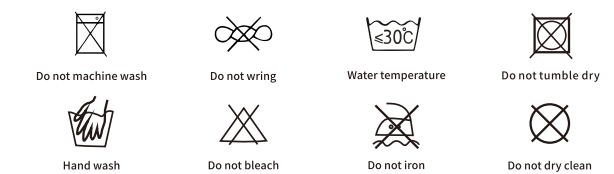

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Sep.15,2022

[Modification Date] : Jul.15,2023

www.velpeau.com