

VELPEAU®



CMC
Thumb Brace
VP0907

[Product Name] : CMC Thumb Brace

[Product Code] :

- VP0907-SL VP0907-ML VP0907-LL
- VP0907-SR VP0907-MR VP0907-LR
- VP0907C-SL VP0907C-ML VP0907C-LL
- VP0907C-SR VP0907C-MR VP0907C-LR




[Product Specifications] :

Size	Product Code	Left/Right	Colour	Palm Circumference(inch)
S	VP0907-SL	Left	Blue	6"-7 1/2"
	VP0907-SR	Right	Blue	
	VP0907C-SL	Left	Black	
M	VP0907C-SR	Right	Black	7 1/2"-9"
	VP0907-ML	Left	Blue	
	VP0907-MR	Right	Blue	
L	VP0907C-ML	Left	Black	9"- 10"
	VP0907C-MR	Right	Black	
	VP0907-LL	Left	Blue	
	VP0907-LR	Right	Blue	
	VP0907C-LL	Left	Black	
	VP0907C-LR	Right	Black	

www.velpeau.com

[Size Guide] :

- Measure your Palm circumference.
- Choose the appropriate size from the chart based on your measurement.

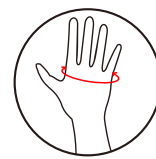

[Product Features] :

- The product features a flexible metal strip that surrounds the base of the thumb and can be customized to fit comfortably. It provides excellent support without hindering normal thumb movement.
- Two-color design molding is used for the main part, utilizing high-strength plastic material for optimal support. The edges are wrapped with soft rubber to prevent skin irritation.
- The inclusion of a high-elastic thumb sleeve enhances comfort and prevents direct contact between the product and the skin. It absorbs sweat, allows air flow, and provides protection against abrasions. The thumb sleeve can also be used separately for low-strength support when needed.
- The strap is freely replaceable, making it easy to put on and take off. The product is suitable for use during various activities such as work, entertainment, and leisure.



Included: Elastic Thumb Support Adjustable metal strip for a custom fit

[Product Comounds] :

TPU Plastic, Stainless Steel, Nylon. (Elastic Thumb Support: Nylon, Spandex)

www.velpeau.com



[Indications] :

It is used for Osteoarthritis of the first CMC joint, Instability of the first CMC joint, Rhizarthrosis, Arthroplasty of the first CMC joint and Post-surgical treatment of the first CMC joint.

[Warning] :

- Do not use this product on the skin that has skin diseases, open wounds, or injuries in the application area.
- This product is intended for self-use only and should not be shared.
- Consult with a healthcare professional before using this product in the following cases:
 1. Blood circulation and sensory disorders in the wrist area, such as diabetes.
 2. Arterial vascular diseases (Arterial occlusive disease).
 3. Venous vascular diseases, such as phlebitis and venous thrombosis.
 4. Disorders of the lymphatic system in the arms.
 5. Allergic reactions to any components of the product.
 6. Specific morphological peculiarities in the wrist being treated may restrict or complicate treatment with the support and its proper positioning.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue immediately and consult a healthcare professional.

www.velpeau.com

VELPEAU®

Shenzhen Weprotex Technology Product Co.,Ltd
Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China
Tel: 0755-89505533 E-mail: sales88@velpeau.com
www.velpeau.com

EC REP: No. 35, Boulevard Renard Banoiti, 75680 Epone, France Made in China



[Wear Steps] :



⚠ After putting on the product, relax your thumb and forms an "O" shape with the index finger (As shown in Fig.5 above), squeeze the metal strip inside the product with the other hand, and knead it to adjust to the fit you want(As shown in Fig.6 above)

www.velpeau.com

[Instructions] :

- Please choose the appropriate size, position the product on the corresponding area, and adjust the straps to ensure comfort and relieve pressure on the affected area. Then, securely fasten the straps.
- Achieving a firm and comfortable compression is recommended for optimal results.
- When applying the product, please follow these instructions:
 - i. Place the orthosis on the thumb, ensuring that the two straps are positioned across the back of the hand (dorsal side).
 - ii. While keeping the fingers in a functional position, secure the lower strap first and then the upper strap.

[Important Factors] :


- If experience any discomfort during the use of the product, please adjust the tightness according to optimal comfort and support.
- This product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.

[Transportation And Storage Conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

[Washing] :

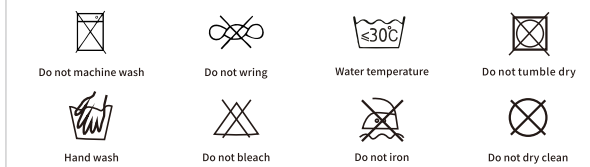

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Nov.9,2021

[Modification Date] : Jul.15,2023

www.velpeau.com