



[Product Name] : Thumb Brace (Breathable Version)

[Product Code] :

□ VP0911-S □ VP0911-M □ VP0911-L

[Product Specifications] :

Size	Product Code	Wrist Circumference (inch)	Left/Right
S	VP0911-S	5 1/3"-6 1/3"	Fits both
M	VP0911-M	6 1/3"-7 1/3"	
L	VP0911-L	7 1/3"-8 1/3"	

[Product Features] :

- Utilizes an ultrasonic production technology to minimize skin irritation caused by friction.
- Hollow design, the product ensures all-around ventilation and air permeability for enhanced comfort.
- Incorporates a high elastic nylon support that conforms to the curve of the thumb, providing complete wrapping and stabilization of the first metacarpophalangeal joint and the base of the thumb.
- Allows unrestricted movement of the remaining fingers due to its low-profile design.
- Includes a high elastic thumb sleeve that can be used in conjunction with the product to prevent direct skin contact, absorb sweat, promote breathability, and prevent abrasions. The thumb sleeve can also be used independently for mild support when minimal strength is needed.
- Easy to wear and removable.
- Suitable for both left and right hands.

www.velpeau.com



- The elastic thumb support prevents uncomfortable irritation, friction, and pinching by creating a barrier between the support brace (splint) and the skin.
- Suitable for standalone use, providing gentle support on its own.



- The highly elastic nylon splint is specifically shaped to fit the contours of the thumb, providing comfortable and flexible support.

[Product Compounds] :

Polyester, Polyurethane, Cotton, Nylon, TPU.

www.velpeau.com

[Application] :

- Ideal for individuals experiencing discomfort or pain in the thumb, offering comfortable protection and moderate support.
- Suitable for various conditions, including soft tissue injuries, ligament strains, gamekeeper's thumb, osteoarthritis, and degenerative joint disease.
- Beneficial for those with arthritic symptoms or pain caused by repetitive stress activities like texting and gaming.

[Precautions / Warnings] :

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

www.velpeau.com




[Instructions] :

- Please choose the appropriate size, position the product on the corresponding area, and adjust the straps to ensure comfort and relieve pressure on the affected area. Then, securely fasten the straps.
- It is important to check the strap tightness to ensure stability. Avoid tightness, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop fasteners.

[How to wear] :



www.velpeau.com

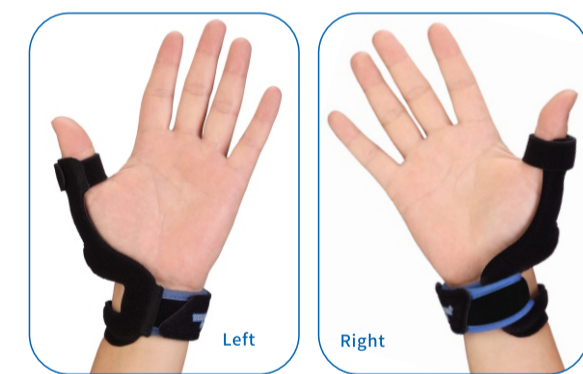


[Transportation And Storage Conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

[Washing] :

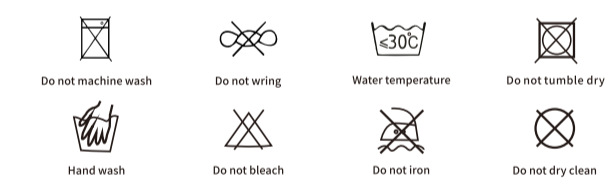

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755 - 89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Jun. 9, 2022

[Modification Date] : Jul. 15, 2023

www.velpeau.com