

VELPEAU®



Black
VP1002



Khaki
VP1002B



[Product Name] : Hernia Belt (Double Inguinal Hernia Truss)

[Product Code] :


[VP1002]
 VP1002-S VP1002-M VP1002-L

[VP1002B]
 VP1002B-S VP1002B-M VP1002B-L

[Product Specifications] :

Size	Product code	Color	Hip circumference(inch)
S	VP1002-S	Black	30"- 36"
	VP1002B-S	Khaki	
M	VP1002-M	Black	35"- 42"
	VP1002B-M	Khaki	
L	VP1002-L	Black	41"- 46"
	VP1002B-L	Khaki	

Please measure your hip using a soft tape measure and select the appropriate size based on the measurement.

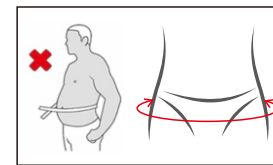

[Product Compounds] :

Nylon, Filament, Polyester, Spandex, Cotton, Low Melt Yarn, Rubber, Polyether, Modal fabric, etc.

www.velpeau.com

[Product Features] :

- Utilizes high-tech fabrics that provide a snug fit without being overly tight in the crotch area, ensuring breathability and preventing sweat build-up.
- Offers secure and firm stability, maintaining its position even during extended wear. The high-quality Hook and loop straps used in the product ensures strong adhesion without causing damage to clothing, and it has a service life of over 1000 uses.
- Features removable triangle compression pads made of medical-grade polyester material. These pads are highly elastic without causing constriction, providing constant and mild pressure for single or double hernias. The pads can be easily adjusted and attached in place using the hook and loop strap.
- Designed with simplicity and fashion which makes it easy to put on and take off.
- Does not include any hard fittings or plastic buckles, ensuring a comfortable fit. The product is suitable for wearing under clothes during daily activities, exercise, and even while sleeping.
- It maintains its shape and elasticity even after long-term use, without crimping or losing elasticity.
- Allows for unrestricted use of the restroom and does not hinder daily activities.
- Suitable for individuals of all genders and age groups, including teenagers, middle-aged, and older individuals.

[Application] :

- Inguinal hernia.
- Femoral hernia.
- Use after and before hernia surgery.

[WARNING] :

- Avoid using this product if you have blisters, erythema, hydrocele, or any injured skin conditions.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue use immediately and consult a healthcare professional.

www.velpeau.com




VELPEAU®

[Important Note] :

- If you experience discomfort during use, please adjust the tightness of the hernia belt.
- Always ensure that the hernia cushion is in close contact with the hernia gap to prevent protrusion of the hernia contents.
- If the hernia protrude from the gap, gently push them back and adjust the position and tightness of the hernia belt.
- Ideally, wear the hernia belt 24 hours a day, including during bathing and using the bathroom.
- If the hernia contents are no longer protruding but the hernia gap has not fully healed, continue wearing the belt to aid in consolidation.
- The effectiveness of the hernia belt may vary depending on individual constitution and the duration of use. Typically, continuous use for more than three months is recommended.
- This product is designed to be reusable. Please wash the product with water before or after each use and store it in a well-ventilated, dry place at room temperature.

[Product component] :


1. Removable compression pads
2. Breathable waist belt
3. Compression straps
4. Hook and loop

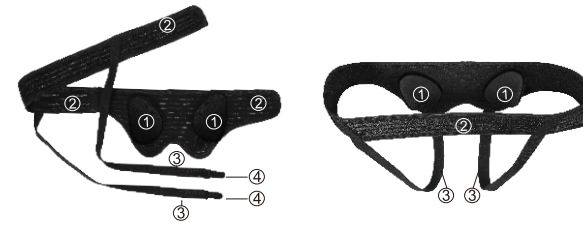

www.velpeau.com

[Instruction & How to wear] :

To ensure proper placement, the compression pads should be positioned with the larger end facing downward and the straight edge facing outward.

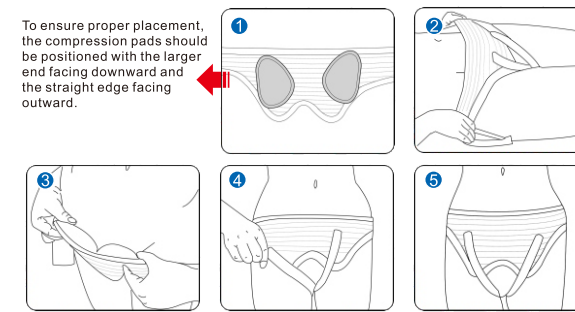

1. Open the belt and install the hernia compression pads. Note: The installation direction of the compression pads must be with the larger end of the compression pads facing downward and the straight edge of the pads facing outward. (Figure 1)
2. Please lie down on your back (supine position) and gently push the hernia back into the abdomen before wearing the hernia belt. (Figure 2)
3. Place the hernia compression pad against the hernia and fasten the waist belt securely. (Figure 3)
4. Wrap the two compression straps from the back to the front of the hernia belt (avoiding the scrotum and testis) and fasten them with the hook and loop. (Figure 4)
5. Adjust the tightness of the waist belt and two compression straps, ensuring that the compression pads are securely supported and do not move away during physical activity. (Figure 5)

www.velpeau.com



Scan the code to watch the wearing steps




[Transportation And Storage Conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

[Washing] :

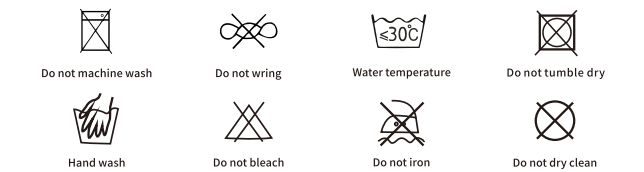

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Nov.19, 2020

[Modification Date] : Jul.15, 2023

www.velpeau.com

Shenzhen Weprotex Technology Product Co., Ltd
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China
 Tel: 0755-89505533 E-mail: sales88@velpeau.com
 www.velpeau.com
 EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France Made in China