

# VELPEAU®



Umbilical  
Hernia Belt  
VP1004

Black  
VP1004



Khaki  
VP1004B



Scan the QR code  
for detailed product information.

www.velpeau.com

**[Product Name]** : Umbilical Hernia Belt

**[Product Code]** :

**[VP1004]**

VP1004-S     VP1004-M     VP1004-L


**[VP1004B]**

VP1004B-S     VP1004B-M     VP1004B-L

**[Product Specifications]** :

Size	Product code	Color	Waist circumference(inch)
S	VP1004-S	Black	<35 1/2"
	VP1004B-S	Khaki	
M	VP1004-M	Black	35 1/2"-40 1/2"
	VP1004B-M	Khaki	
L	VP1004-L	Black	40 1/2"-50"
	VP1004B-L	Khaki	

Please measure your waist at the level of the umbilicus (belly button) using a soft tape measure, and select the appropriate size based on the measurement.

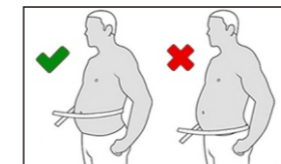

**[Product Compounds]** :

Nylon, Filament, Polyester, Spandex, Low Melt Yarn, Polyether, Cotton, etc.

www.velpeau.com

# VELPEAU®



**Shenzhen Weptex Technology Product Co.,Ltd**  
Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China  
Tel: 0755 - 89505533    E-mail: sales88@velpeau.com  
www.velpeau.com  
EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France    Made in China

**[Product Features]** :

- The compression belt utilizes elongated mesh fabric with perforated hernia pads, ensuring excellent ventilation and preventing sweat accumulation.
- The anterior fastening method is user-friendly, addressing the inconvenience of posterior fastening.
- The hernia pad is made from medical-grade polyether material. It is lightweight and mimics the elasticity of human muscles, gently and comfortably aiding in hernia reduction.
- The product is crafted from neoprene-free and latex-free components, ensuring it is skin-friendly and non-irritating.

**[Application]** :

- Umbilical hernia.
- Incisional hernia.
- Ventral hernia.

**[Important Note]** :


- If you experience discomfort during use, please adjust the tightness of the hernia belt.
- Always ensure that the hernia cushion is in close contact with the hernia gap to prevent protrusion of the hernia contents.
- If the hernia protrude from the gap, gently push them back and adjust the position and tightness of the hernia belt.
- Ideally, wear the hernia belt 24 hours a day, including during bathing and using the bathroom.
- If the hernia contents are no longer protruding but the hernia gap has not fully healed, continue wearing the belt to aid in consolidation.
- The effectiveness of the hernia belt may vary depending on individual constitution and the duration of use. Typically, continuous use for more than three months is recommended.
- This product is designed to be reusable. Please wash the product with water before or after each use and store it in a well-ventilated, dry place at room temperature.

www.velpeau.com

**[WARNING]** :

- Avoid using this product if you have blisters, erythema, or any injured skin conditions.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue use immediately and consult a healthcare professional.

**[How to wear]** :

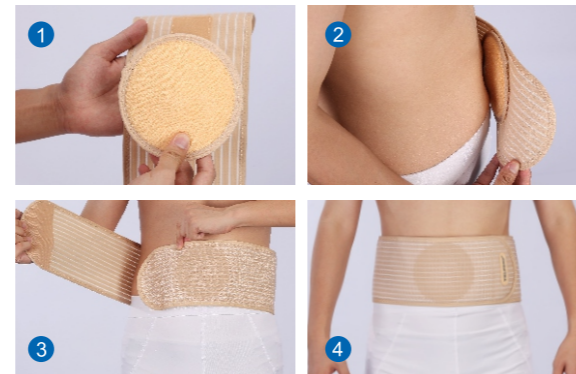

- 1.Unfasten the belt and attach the hernia pad to the hook and loop on the end of the belt to latch together. (Figure ①)
- 2.Align the hernia pad on the hernia and gently push it from the hernia pad back into the abdomen. (Figure ②)
- 3.Then fasten the belt securely. (Figure ③)
- 4.Adjust the tightness of the compression belt so that the hernia no longer protrudes during movement. (Figure ④)


www.velpeau.com



**[Transportation And Storage Conditions]** :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

**[Washing]** :

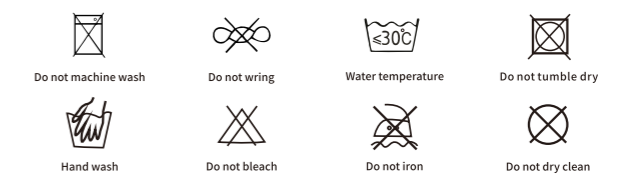

**[Term of Validity]** : 5 years

**[Production Date]** : See the package

**[Manufacturer]** : Shenzhen Weptex Technology Product Co., Ltd.

**[Production Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service]** : Shenzhen Weptex Technology Product Co., Ltd.

**[After-sale Service Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode]** : 518129

**[Tel]** : +86 - 755-89505533

**[E-mail]** : sales88@velpeau.com

**[Website]** : www.velpeau.com

**[Date of Preparation]** : Aug.15,2023

**[Modification Date]** : Aug.15,2023

www.velpeau.com