


70mm

110mm



**VELPEAU®**

FDA  
CE  
ISO 1309901  
ISO 13013485

**Hernia Belt VP1005**  
Umbilical Hernia Belt for Infant and Toddler

**[Product Name] :**  
Hernia Belt(Umbilical Hernia Belt for Infant and Toddler )

**[Product Code] :** VP1005

**[Product Specifications] :**

Size	Product Code	Colour	Waist circumference(inch)
ONE SIZE FITS ALL	VP1005	Khaki	<20"


**[Product Comounds] :**  
Nylon, Filament, Polyester, Spandex, Low Melt Yarn, Polyether, Cotton, etc.


**[Application] :**

- Conservative treatment of umbilical and paraumbilical hernias and epigastric hernia.
- Prolapse and incarceration of hernial sac and other complications.
- Prevention of relapses after herniotomy.

www.velpeau.com

**[Product Components] :**

**Front** 

**Back** 


This product includes three different shapes of hernia pads, each designed for different stages of recovery. To ensure optimal support, please consult with a doctor and select the appropriate pad based on the recovery phase:

**Early Stage (Days 1-5):**  
Begin with the concave oval-shaped hernia pad. (While using this pad, there will be a significant reduction in protrusion after use)

**Mid-Stage (Days 6-15):**  
Switch to the convex oval-shaped hernia pad during this stage. (There will be a noticeable decrease in protrusion as the recovery progresses)

**Last Stage (Days 16 - Per Physician's Advice):**  
Use the circle-shaped hernia pad as the final phase of recovery.

www.velpeau.com




**[Product Features] :**

- The compression belt utilizes elongated mesh fabric with perforated hernia pads, ensuring excellent ventilation and preventing sweat accumulation.
- Three different shapes of hernia pads (concave oval, convex oval, and circle-shaped) are included with this product, allowing for selection based on the severity of hernia protrusion and the specific needs of infants or toddlers during their recovery process.
- The hernia pad is made from medical-grade polyether material. It is lightweight and mimics the elasticity of human muscles, gently and comfortably aiding in hernia reduction.
- The product is crafted from neoprene-free and latex-free components, ensuring it is skin-friendly and non-irritating.

**[WARNING] :**

- Use with caution if the infant or toddler has sensitive skin. DO NOT use this product if the infant or toddler has irreducible herniated contents or experiences hernia rupture and swelling in the hernia area.
- In very rare cases, this product may cause skin irritations and allergies. If experience any adverse reactions such as redness, itching, or rash, discontinue use immediately and consult a healthcare professional.

www.velpeau.com




**VELPEAU®**

Shenzhen Weprotex Technology Product Co.,Ltd  
Add:1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangzou Community, Bantian Street, Longgang District, Shenzhen, China  
Tel: 0755-89505533 E-mail: sales88@velpeau.com  
www.velpeau.com  
EC REP: No. 38, Boulevard Renard Benoit, 78890 Epone, France Made in China

**[How to wear] :**

- Unfasten the compression belt, then choose the appropriate hernia pad (concave oval, convex oval or circle-shaped) to attach with the hook and loop at one end of the belt.
- IMPORTANT:** It is highly advisable to consult with your doctor to ensure you select the correct hernia pad.
- Lay the infant or toddler flat on their back, gently massage any protruding umbilical hernia back into the abdomen, then position the hernia pad over the hernia and press it firmly.
- Wrap the compression belt around the infant or toddler's belly, fasten it snugly and securely. Make sure it's not too tight or too loose, just enough to prevent the hernia from popping out. While wearing it, regularly check if the hernia pad is securely covering the hernia area.
- If the belly button hasn't fully healed, it's recommended to use medical cotton (gauze) pads to cover the belly button before using this product. This hernia belt should be worn for 3 to 24 hours a day unless your doctor prescribes a different schedule.




www.velpeau.com

**[Important Note] :**


- If you experience discomfort during use, please adjust the tightness of the hernia belt.
- Always ensure that the hernia cushion is in close contact with the hernia gap to prevent protrusion of the hernia contents.
- If the hernia contents protrude from the gap, gently push them back and adjust the position and tightness of the hernia belt.
- Ideally, wear the hernia belt 24 hours a day.
- If the hernia contents are no longer protruding but the hernia gap has not fully healed, continue wearing the belt to aid in consolidation.
- The effectiveness of the hernia belt may vary depending on individual constitution and the duration of use.
- This product is designed to be reusable. Please wash the product with water before or after each use and store it in a well-ventilated, dry place at room temperature.

**[Transportation And Storage Conditions] :**  
The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

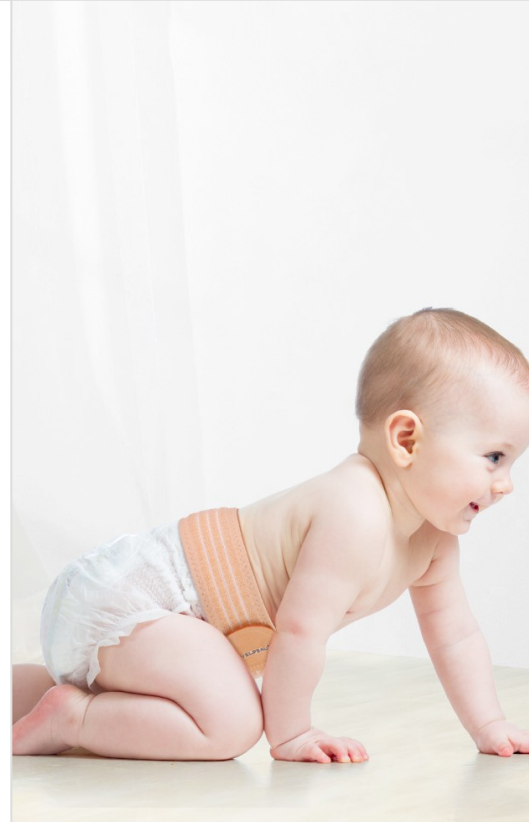

**[Washing] :**

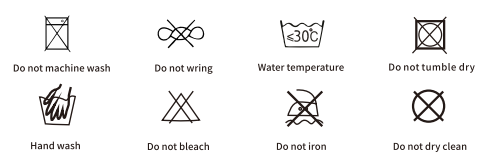

**[Term of Validity] :** 5 years  
**[Production Date] :** See the package  
**[Manufacturer] :** Shenzhen Weprotex Technology Product Co., Ltd.  
**[Production Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangzou Community, Bantian Street, Longgang District, Shenzhen, China  
**[After-sale Service] :** Shenzhen Weprotex Technology Product Co., Ltd.  
**[After-sale Service Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangzou Community, Bantian Street, Longgang District, Shenzhen, China  
**[Postcode] :** 518129  
**[Tel] :** +86 - 755-89505533  
**[E-mail] :** sales88@velpeau.com  
**[Website] :** www.velpeau.com  
**[Date of Preparation] :** Oct.26,2023  
**[Modification Date] :** Oct.26,2023