

VELPEAU®



[Product Name] : Sacroiliac Belt


[Product Code] : VP1021B

VP1021B-Reg VP1021B-XL/XXL

[Product Specifications] :

Size	Hip circumference (inch)	Belt length (inch)	Belt width (inch)
Reg	28 1/2"-44"	43"	3 3/4"
XL/XXL	44"-54"	48 1/2"	4 1/2"

Please measure your hip using a soft tape measure and select the appropriate size based on the measurement.

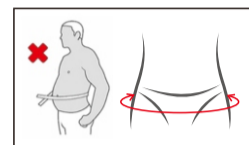

[Product Features] :

- Utilizes lightweight high-tech fabrics that provide a snug fit, ensuring breathability and preventing sweat build-up.
- Offers secure and firm stability, maintaining its position even during extended wear. The high-quality Hook and loop straps used in the product ensures strong adhesion without causing damage to clothing, and it has a service life of over 1000 uses.
- Designed with simplicity and fashion which makes it easy to put on and take off.
- Does not include any hard fittings or plastic buckles, ensuring a comfortable fit. The product is suitable for wearing under clothes during daily activities, exercise, and even while sleeping.
- It maintains its shape and elasticity even after long-term use, without crimping or losing elasticity.
- The product is neoprene and latex-free, ensuring no skin irritation.
- Suitable for individuals of all genders and age groups, including teenagers, middle-aged, and older individuals.

www.velpeau.com

[Application] :

- Improves core strength & increases mobility. Unisex.
- Pain related to the sacroiliac joint and sciatic nerve.
- Pain in the lower back, buttocks, pelvic region, and thighs.
- Symphysis pubis dysfunction (also known as pelvic girdle pain).
- Piriformis muscle injury and piriformis syndrome.
- Pelvic tilt (anterior or posterior).



[Product Compounds] :

Nylon, Filament, Polyester, Spandex, Low Melt Yarn, Polyether, Cotton, etc.

[Instructions] :

- Stand upright and position the first elastic band horizontally above the pubis.
- Position the second elastic band horizontally over the first elastic band.
- Ensure that the strap is neither too tight nor too loose to maintain stability after wearing.
- It is advisable to wear the product over jeans or fabric with an uneven surface to prevent slipping.
- Not recommended for use while sitting down.

www.velpeau.com

VELPEAU®




Shenzhen Weprotex Technology Product Co., Ltd

Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China
Tel: 0755-89505533 E-mail: sales88@velpeau.com
www.velpeau.com


EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France Made in China




[How to wear] :

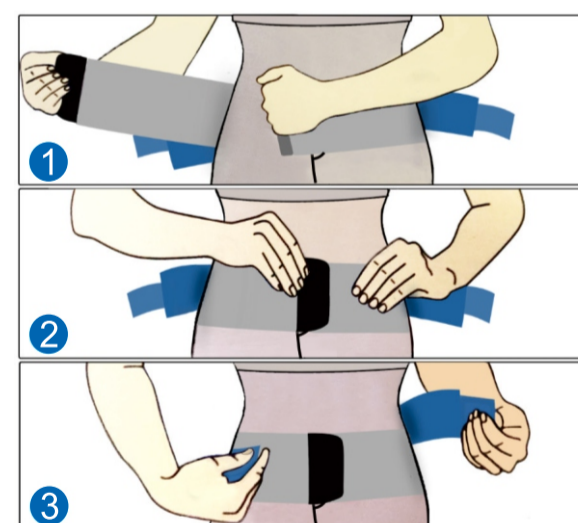

[Wear Position] :

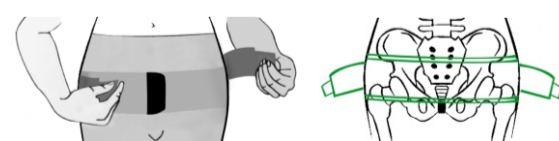

www.velpeau.com

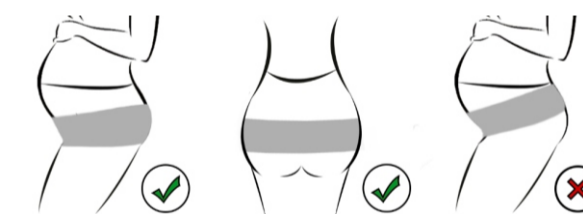

[Precautions/ Warnings] :

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.



Scan the QR code for detailed product information.

www.velpeau.com

[Transportation and storage conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

[Washing] :



[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Oct.10,2023

[Modification Date] : Oct.10,2023

www.velpeau.com