

# VELPEAU®



Knee Immobilizer VP1202

[Tri-panel Version]



**[Product Name]** : Knee Immobilizer ( Tri-panel )

**[Product code]** :

VP1202-S    VP1202-M    VP1202-L

**[Product Specifications]** :

Size	Product code	Product length (inch)	Left/Right
S	VP1202-S	16"	Fits both
M	VP1202-M	20"	
L	VP1202-L	24"	

**[Product Features]** :

- Designed to set the joint at 0° to 20°, providing optimal leg straightening and suitability for wearing during sleep.
- The tri-panel design allows for flexible adjustment of the position of the fixed support bar on both sides, catering to individuals with different leg sizes.
- The hollow patella design allows for easy wound care and facilitates wearing and removal.
- The four hook and loop straps provide strong stability from different directions and can be adjusted to accommodate various leg circumferences.
- High-strength removable aluminum alloy protectors are strategically placed on the inner side, outward side, and backward side, providing firm support and stability.
- The product is filled with lightweight and soft sponge material, which is skin-friendly, breathable, and enhances safety and comfort.
- The shoulder strap effectively prevents the support from sliding during movement.

www.velpeau.com

# VELPEAU®



**Shenzhen Weptrex Technology Product Co., Ltd**  
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755-89505533   E-mail: sales88@velpeau.com  
 www.velpeau.com  
 EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France   Made in China

**[Product Compound]** :

Polyester, Polyurethane, Cotton, Nylon, Polypropylene, Aluminum alloy.

**[Instructions]** :

- Proper sizing can be achieved by following the wear step diagram in the manual.
- It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop (Velcro) fastener.
- It is recommended to wear soft leggings or pajama pants underneath the product to prevent it from sliding easily during walking.

**[Precautions/ Warnings]** :

- Please consult a professional for guidance before using this product.
- This product is intended for adult and adolescent use only.
- It is designed for easy use, either independently or with the assistance of medical personnel or others when needed.
- If any discomfort occurs during use, adjust the tightness accordingly.
- The product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.
- This product is designed for standalone use and should not be combined with other products. Its safety and effectiveness are not affected by external conditions.
- No specific special skills, knowledge, or training required for users. The product is made from materials that comply with the ROHS standard which is free from allergens or side effects on the human body.
- If there is an open wound on the skin, please consider bandaging the wound to prevent direct contact.

www.velpeau.com



**[Application]** :

- Preoperative and postoperative fixation of knee joint.
- Patellar dislocation.
- Knee ligament injury.
- After knee arthroscopy surgery.

**[How to wear]** :




www.velpeau.com

**[Transportation and storage conditions]** :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

**[Washing]** :

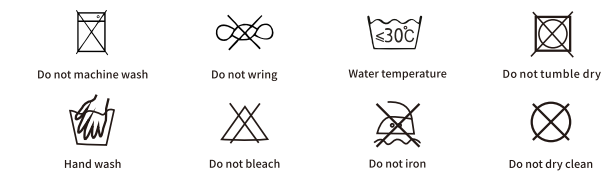

**[Term of Validity]** : 5 years

**[Production Date]** : See the package

**[Manufacturer]** : Shenzhen Weptrex Technology Product Co., Ltd.

**[Production Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service]** : Shenzhen Weptrex Technology Product Co., Ltd.

**[After-sale Service Address]** : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode]** : 518129

**[Tel]** : +86 - 755 - 89505533

**[E-mail]** : sales88@velpeau.com

**[Website]** : www.velpeau.com

**[Date of Preparation]** : Jan.19,2017

**[Modification Date]** : Jul.15,2023

www.velpeau.com