

# VELPEAU®



**[Product Name] :** Stirrup Ankle Brace ( Ankle Splint )

**[Product Code] :**

- VP1501-SL    VP1501-SR    VP1501-LL    VP1501-LR
- VP1501B-SL    VP1501B-SR    VP1501B-LL    VP1501B-LR

**[Product Specifications] :**

Size	Version	Product code	Body height(inch)	Left/Right
S	Foam pads	VP1501-SL	<63"	Left
		VP1501-SR		Right
	Gel pads	VP1501B-SL		Left
		VP1501B-SR		Right
L	Foam pads	VP1501-LL	≥63"	Left
		VP1501-LR		Right
	Gel pads	VP1501B-LL		Left
		VP1501B-LR		Right

**[Product Features] :**

- Harder outer splint, that is ergonomically designed to tightly wrap the ankle joint, providing uniform pressure and good support.
- Breathable, skin-friendly pads made from high-density sponge (Foam Pads version) or medical liquid gel (Gel Pads version) provide insulation from the hard external splint with excellent thermal conductivity.
- Two adjustable lateral straps ensure quick and secure fastening, offering a skin-friendly, soft feel while maintaining pressure balance and preventing loosening.
- Easily adjustable heel strap accommodates various ankle sizes for a customized fit.
- Easily removable and reusable.

www.velpeau.com

**[Product Compounds] :**

Foam Pads Version: Polypropylene, Polyurethane, Polyester, Nylon.  
Gel Pads Version: Polypropylene, TPU, Gel (Carboxymethyl Cellulose), Nylon.

**[Application] :**

- Conservative treatment and postoperative care of ankle fibular ligament injury (Grade 2 to 3).
- Chronic ankle instability (Habitual sprain).
- After ankle sprain.

**[Instructions] :**

- Choose the correct type and size, ensuring proper distinction between right and left feet.
- Loosen the straps before wearing and place the splint on both sides of the ankle, noting the distinction between the inside and outside then securing both straps firmly.
- Adjust the tightness of the straps and heel strap for optimal comfort and stability.
- For individuals with sensitive skin, it is recommended to wear socks to prevent direct contact with the skin.
- After relieving swelling, the product can be worn with low-top, loose, and soft lace-up shoes. Loosen shoelaces as much as possible before putting it on.

▲ Gel pads can be used for both ice and hot compress.

- Freezing Method:  
Refrigerate the gel pad for 2-3 hours before use.
- Heating Method:  
Heat the gel pad in hot water or microwave.  
If using a microwave, wrap the gel pad in a wet towel and heat on medium or high heat for no more than 30 seconds and no more than 140 °F.

www.velpeau.com

**[Precautions / Warnings] :**

- Do not use this product on the skin that has skin diseases, open wounds, or injuries in the application area.
- This product is intended for self-use only and should not be shared.
- Consult with a healthcare professional before using this product in the following cases:
  1. Blood circulation and sensory disorders in the legs area, such as diabetes.
  2. Arterial vascular diseases (Arterial occlusive disease).
  3. Venous vascular diseases, such as phlebitis and venous thrombosis.
  4. Disorders of the lymphatic system in the legs.
  5. Allergic reactions to any components of the product.
  6. Specific morphological peculiarities in the ankles being treated may restrict or complicate treatment with the support and its proper positioning.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue immediately and consult a healthcare professional.

**[Important Factors] :**

- If experiences any discomfort during the use of the product, please adjust the tightness according to optimal comfort and support.
- This product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.

**[Easy to Take Apart] :**



www.velpeau.com



# VELPEAU®


**[How to wear] :**



Scan the code to watch the wearing steps



**[Product Component] :**

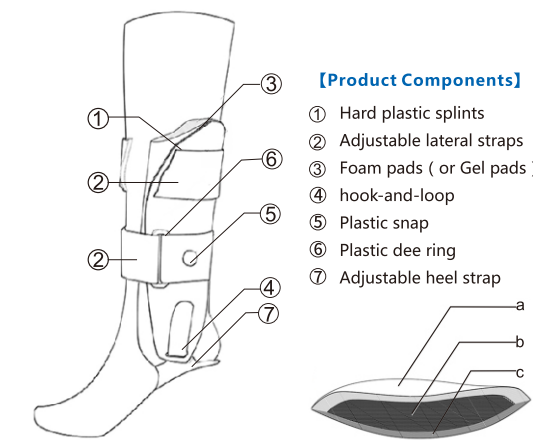

**[Product Components] :**

- ① Hard plastic splints
- ② Adjustable lateral straps
- ③ Foam pads ( or Gel pads )
- ④ hook-and-loop
- ⑤ Plastic snap
- ⑥ Plastic dee ring
- ⑦ Adjustable heel strap

www.velpeau.com

**[Foam Pads] :**

- a. Suede
- b. Sponge
- c. Polyester velvet

Foam pads Version  
VP1501



**[Gel Pads] :**

- a. TPU
- b. Gel ( Garboxymethyl Cellulose )

Gel pads Version  
VP1501B




**[Transportation And Storage Conditions] :**

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

**[Washing] :**

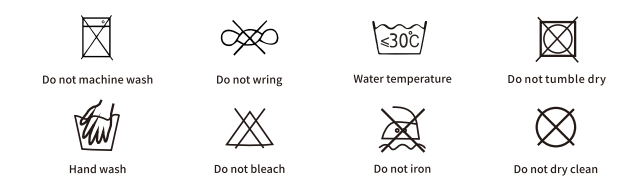

**[Term of Validity] :** 5 years

**[Production Date] :** See the package

**[Manufacturer] :** Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service] :** Shenzhen Weprotex Technology Product Co., Ltd.

**[After-sale Service Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode] :** 518129

**[Tel] :** +86 - 755 - 89505533

**[E-mail] :** sales88@velpeau.com

**[Website] :** www.velpeau.com

**[Date of Preparation] :** Jan. 9, 2017

**[Modification Date] :** Jul. 15, 2023

www.velpeau.com

Shenzhen Weprotex Technology Product Co., Ltd  
Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtuo Community, Bantian Street, Longgang District, Shenzhen, China  
Tel: 0755 - 89505533 E-mail: sales88@velpeau.com  
www.velpeau.com

EC REP: No. 38, Boulevard Renard Benoît, 78680 Epone, France

Made in China