

# VELPEAU®



**[Product Name] :** Stirrup Ankle Brace


**[Product Code] :**

- VP1503-FB     VP1503-FG  
 VP1503-GB     VP1503-GG

**[Product Specifications] :**

Size	Product Type	Product Code	Colour	Foot Length	Left/Right
One Size	Foam Pads	VP1503-FB	Blue	8 1/2"-11 1/2"	Fits both
		VP1503-FG	Grey		
	Gel Pads	VP1503-GB	Blue		
		VP1503-GG	Grey		

**[Measurement Method] :**

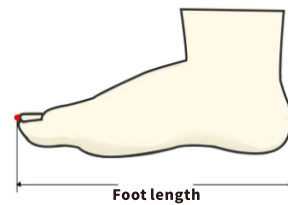

**[Product Compounds] :**

**Foam Pads Version:** Modal fabric, Polyurethane, Polyester, Polypropylene, Thermoplastic Rubber(TPE), Nylon.  
**Gel Pads Version:** Gel(Carboxymethyl Cellulose), Polypropylene, Thermoplastic Rubber(TPE), TPU, Nylon.

www.velpeau.com



**[ Product Features ] :**

- This product is designed to fit all sizes and is suitable for left and right ankle.
- The gel fill-in protector provides a soft and flexible feel to the ankle, ensuring that the ankle is not directly in contact with the hard plastic splint, offering greater comfort.
- The soft gel covering the edge of the plastic splint prevents skin scratches or shoe damage.
- Breathable, skin-friendly pads made from high-density sponge (Foam Pads version), or medical liquid gel (Gel Pads version) provide insulation from the hard external splint with excellent thermal conductivity.
- The adjustable heel strap accommodates different ankle sizes, while the two adjustable lateral straps ensure quick and secure fastening, maintaining a skin-friendly, soft, and balanced pressure to prevent loosening.
- This product is stylish, portable, and easy to carry and store.

www.velpeau.com

# VELPEAU®



**Shenzhen Weprotex Technology Product Co.,Ltd**  
 Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China  
 Tel: 0755-89505533    E-mail: sales88@velpeau.com  
 www.velpeau.com  
 EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France    Made in China

**[Instructions] :**

- Take off the foam (gel) pads on the splint before wearing and adjust the width of the heel strap for a better fit.
- Sit on a step stool with your lower legs at a 90° angle to the ground, then place your heel on the rounded corners of the heel strap with the arrow on the hard-shell splint pointing toward the toe. Adjust the splints on both sides according to the ankle joint condition, aligning the gel protective fill-in with the prominent positions of the ankle bones, and ensuring an even gap between the splints.
- Pass the upper adjustable strap through the front of the shin and secure it with the buckle. Place the lower adjustable strap on the hook and loop, pass it through the front of the shin, and secure it with the strap.
- It is recommended to wear socks with the product.
- After relieving swelling, the product can be worn with low-top, loose, and soft lace-up shoes. Loosen shoelaces as much as possible before putting it on.

**[Application] :**

- Ankle ligamentous injury(level 1 to level 2).
- After ankle sprain.
- Chronic ankle instability.
- Swelling around the ankle, hydrops articulari, intra-articular inflammation.
- After surgery or after wearing a plaster cast.

⚠ Gel pads can be used for both ice and hot compress.

**Freezing Method:** Refrigerate the gel pad for 2-3 hours before use.

**Heating Method:** Heat the gel pad in hot water or microwave.

If using a microwave, wrap the gel pad in a wet towel and heat on medium or high heat for no more than 30 seconds and no more than 140 °F.

www.velpeau.com

**[How to wear] :**



Scan the code to watch the wearing steps

**[Important Factors] :**

- If experiences any discomfort during the use of the product, please adjust the tightness according to optimal comfort and support.
- This product is designed to be reusable, please wash the product with water before or after each use. Store it in a well-ventilated, dry place at room temperature.

www.velpeau.com

**[Warning] :**

- Do not use this product on the skin that has skin diseases, open wounds, or injuries in the application area.
- This product is intended for self-use only and should not be shared.
- Consult with a healthcare professional before using this product in the following cases:
  1. Blood circulation and sensory disorders in the legs area, such as diabetes.
  2. Arterial vascular diseases (Arterial occlusive disease).
  3. Venous vascular diseases, such as phlebitis and venous thrombosis.
  4. Disorders of the lymphatic system in the legs.
  5. Allergic reactions to any components of the product.
  6. Specific morphological peculiarities in the ankles being treated may restrict or complicate treatment with the support and its proper positioning.
- In very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue immediately and consult a healthcare professional.

**[Product Component] :**

1. Bi-color injection plastic splint
2. Adjustable lateral straps(Upper)
3. Adjustable lateral straps(Below)
4. Foam pads(or Gel pads)
5. hook and loop
6. Seat-belt buckle stop button
7. Plastic dee ring
8. Adjustable heel strap




www.velpeau.com

**[Transportation And Storage Conditions] :**

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

**[Washing] :**

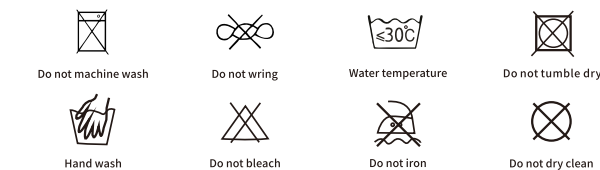

**[Term of Validity] :** 5 years

**[Production Date] :** See the package

**[Manufacturer] :** Shenzhen Weprotex Technology Product Co., Ltd.

**[Production Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

**[After-sale Service] :** Shenzhen Weprotex Technology Product Co., Ltd.

**[After-sale Service Address] :** 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

**[Postcode] :** 518129

**[Tel] :** +86-755-89505533

**[E-mail] :** sales88@velpeau.com

**[Website] :** www.velpeau.com

**[Date of Preparation] :** Sep.18,2017

**[Modification Date] :** Jul.15,2023

www.velpeau.com