# **VELPEAU**®



[Product Name]: Walking Boot (Long Version)

[Product Code]:

□ VP1601-S □ VP1601-M □ VP1601-L

### [Product Specifications]

	Size	Product Code	Shoe size(EUR)	Colour	Left/Right	Product height
	S	VP1601-S	36-39	Grey	Ambidextrous	17"
	М	VP1601-M	39-42			
	L	VP1601-L	42-45			

How To Choose The Right Size?



## [Product Compounds]

PP, EVA, Rubber, Polyurethane, Polyester, Polyester-cotton, Nylon, TPU, Polyethylene foam, POM, etc.

## [Product Features]

- •Ultra-lightweight technology and widened footbed for comfortable wear. •Utilizes durable semi-rigid external shell for effective limb support and comprehensive protection.
- •Advanced anti-slip technology on the outsole reduces the impact on the affected limb during walking. EVA high-rebound shock-absorbing sole
- design mimic's normal gait.

  •Two independent inflatable airbags provide adjustable dynamic compression
- to accelerate swelling reduction.
  •Enclosed toe pad effectively protects the forefoot.
- •Shoe cover features a combination of 3D mesh fabric and sponge for ventilation and comfort, suitable for extended wear. •Suitable for both left and right feet.

— www.velpeau.com —

## [Application] :

- •Stables fractures of the foot, ankle, and lower leg... •Severe ankle sprains.
- •Postoperative immobilization following surgery of the tibiofibular ligaments, soft tissues, and tendons

·Postoperative immobilization.









7-9 lavers









## **VELPEAU**®



enzhen Weprotex Technology Product Co.,Ltd

## [How to wear]











Scan the code to watch the wearing steps

— www.velpeau.com —







— www.velpeau.com -

## [Inflation and Deflation Demonstration]







Please press and hold the area



## [Instructions]

•Please consult a professional for guidance before using this product.
•Proper sizing can be achieved by following the wear step diagram in the manual or watching the detailed wearing video on the official website www.velpeau.com.

•It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support. •The product is designed for easy assembly and disassembly, requiring

simple attachment or detachment of the hook and loop fasteners.

## [Warning] :

•Do not use this product on the skin that has skin diseases, open wounds, or injuries in the application area.

- This product is intended for self-use only and should not be shared.
  Consult with a healthcare professional before using this product in the
- 1. Blood circulation and sensory disorders in the legs area, such as
- 2. Arterial vascular diseases (Arterial occlusive disease).
- Nemous vascular diseases, such as phlebitis and venous thrombosis.
   Disorders of the lymphatic system in the legs.
- S. Allergic reactions to any components of the product.
   Specific morphological peculiarities in the ankles being treated may
- restrict or complicate treatment with the support and its proper positioning.

very rare cases, this product may cause skin irritations and allergies. If you experience any adverse reactions such as redness, itching, or rash, discontinue immediately and consult a healthcare professional

## [Transportation and Storage Conditions]

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

— www.velpeau.com ——

## [Washing]



Hand wash









Do not dry clear

[Production Date]: See the package [Manufacturer]: Shenzhen Weprotex Technology Product Co., Ltd.

X

[Production Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District,

[After-sale Service]: Shenzhen Weprotex Technology Product Co., Ltd. [After-sale Service Address]: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen,

[Postcode]:518129 [Tel]: +86 - 755-89505533

[Term of Validity ]: 5 years

[E-mail]: sales88@velpeau.com [Website]: www.velpeau.com [Date of Preparation]: Aug.1,2023

[Modification Date ]: Aug.1,2023

www.velpeau.com -