

biopen[®]




User Manual **EN**

Please read the manual before use



| | |
|--------------------------|----|
| Product components | 2 |
| Warning | 3 |
| Advantage | 4 |
| Basic Operation | 5 |
| Basic Operation | 6 |
| Basic Operation | 7 |
| Basic Operation | 8 |
| Specification | 9 |
| Maintenance | 9 |
| Warranty | 9 |
| Package list | 9 |
| Precaution | 10 |

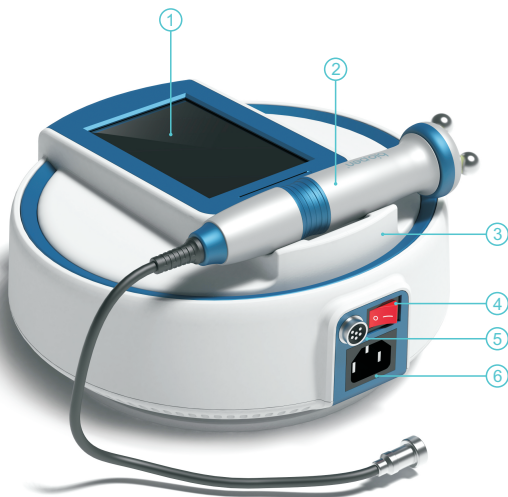

① Touch screen

② Handpiece

③ Handpiece bracket

④ Power switch

⑤ Handpiece socket

⑥ Power socket



Thanks for using our company's products, in order to fully make use of the products, We sincerely suggest you to pay attention to the following instructions before use:

Please read the instruction manual carefully and keep it for further reading and consultation.

Please follow the instructions to properly install and operate the machine.

Please do not disassemble and replace any accessories of the machine.

Opening and removal of the console by anyone other than authorized company personnel is prohibited.

Except for professionals, it is forbidden to replace and maintain the machine. If any problem occurs during operation, please contact us.



The device has both radio frequency(RF)and microcurrent(EMS)technologies.

RF is a skin heating therapy that stimulates collagen regeneration, reduces wrinkles and firms the skin.

EMS enhance the activity of cells, promote muscle exercise, and restore skin elasticity.It accelerate the blood circulation of capillaries and promote the absorption of nutrients in skin tissue.

The 360-degree rotating handpiece can be rotated forward or reverse, or it can be rotated alternately.

The rotation speed is adjustable. 360 degree rotating massage can help fat movement and body shaping.

Combine with blue light phototherapy to achieve better skin care results and get multi-functional treatment.

- Light therapy and rotating RF


- Light therapy and rotating EMS

Safer: non-invasive, does not burn the skin.

Easier: reduce labor consumption of operators.


More comfortable: different rotation modes, smooth and comfortable.

More Efficient: RF and EMS, and combination of rotation and light therapy.

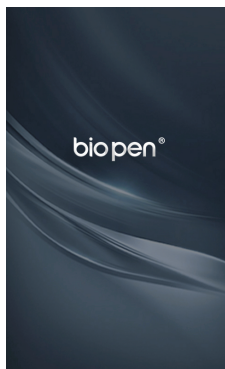

[Fig 1]

1. Connect accessories well to the host .
[Connect the handpiece to the handpiece socket of host and connect the adapter to the power socket of host]
- 2.Plug DC adapter to AC outlet.
- 3.Turn on the power switch to enter boot interface as [Fig 1].
- 4.Click the boot interface [Fig 1] to enter the home interface as [Fig 2] .
- 5.It will show below interface as [Fig 3] when you use EMS function .

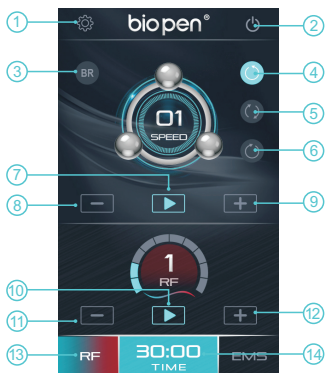

[Fig 2]


- ① Settings
- ② Shutdown
- ③ Blue Ray
- ④ Reverse rotation
- ⑤ Alternate rotation
- ⑥ Forward rotation
- ⑦ Rotation start/pause
- ⑧ Rotation speed down
- ⑨ Rotation speed up
- ⑩ RF/EMS start/pause
- ⑪ RF/EMS intensity down
- ⑫ RF/EMS intensity up
- ⑬ RF mode
- ⑭ Working time
- ⑮ Discontinuous pulse
- ⑯ Continuous pulse
- ⑰ EMS mode



[Fig 3]




6. Please use suitable mode according to related treatment. Selection for RF&EMS.

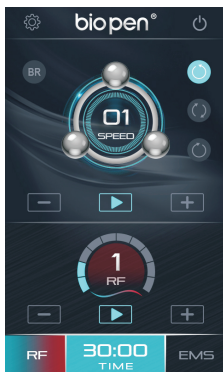

[Fig 4]

RF Mode:

[1] Click  to select the RF mode.
In interface as [Fig 4] .

[2] Click   to start or pause the RF treatment.

[3] Adjust the RF intensity :
Adjust the RF intensity according to different clients and different treatment sites.

The system default RF intensity is 1.

Click  or  to adjust.

The range is from 1 (weak) to 5 (strong).


Notice:

The intensity of RF can be adjusted only after starting the RF.





[Fig 5]

EMS Mode:

[1] Click  to select the EMS mode.
In interface as [Fig 5] .

[2] Click   to start or pause the EMS treatment.



[3] Click  or  to choose the mode of EMS pulse.



This button is for mode of discontinuous.
The pulse of EMS will output discontinuously.



This button is for mode of continuous.
The pulse of EMS will output continuously.

[4] Adjust the EMS intensity : Adjust the RF intensity according to different clients and different treatment sites. Click  or  to adjust. The range is from 1 (weak) to 5 (strong).

Notice:The intensity of EMS can be adjusted only after starting the EMS.

7.Operation of Rotation

[1] Click  or  or  to choose the mode of rotation.





This button is for mode with rotation reverse.





This button is for mode with rotation forward and reverse alternately.



This button is for mode with rotation forward.

[2] Click   to start or pause the rotation.

[3] Adjust the speed of rotation : Adjust the rotation speed according to different clients and different treatment sites. Click  or  to adjust. The range is from 1 (slow) to 5 (fast).


Notice:The speed of rotation can be adjusted only after starting the rotation.

8.Operation of light therapy

[1] Click  to start or shutdown the therapy of blue ray.



[Fig 6]

Click  to enter the setting interface as [Fig 6].

Set default parameter for each function. Please set default parameter for each level of rotation speed & RF & EMS and working time according to your need.


[1] Select the level for corresponding mode or working time.

Notice:
The default working time is 30 minutes.
The maximal working time is 99 minutes.

[2] Click  or  to adjust the parameter.


[3] Click  to finish adjusting parameter.


[4] Press and hold the  button for 4-5 seconds if you want to reset all parameter.

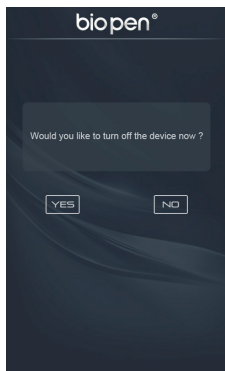

[Fig 7]

Turn off the machine

Click  to enter the interface as [Fig 7] .

Click  to turn off the machine.



- 【Model name】 : bio pen T6
- 【Weight】 : 2.8 kg
- 【Adapter】 : 100V~240V
- 【Size】 : 320*280*150 mm
- 【Patented product】 : Yes
- 【Purpose of Use】 : Skin therapy

Maintenance

[1] General maintenance

- Ensure the device is turned off when not in use. Wipe its surface with a dry cloth.
- Keep it away from direct sunlight or high temperature or humidity.
- Do not leave the measurement, repair of the device to non-professional staffs.

[2] Routine inspection

- Make sure to inspect the device and adapter before use.
- When you use the device which has not been used for a long time, make sure to check if it works properly with safety.

Warranty

The warranty period of the device is 12 months. We offer repair service from the purchase date. If any of our products has damages or natural trouble, we provide replacements or repair service.

Package list

| | |
|-----------|------|
| Host | 1 pc |
| Handpiece | 1 pc |
| Adapter | 1 pc |
| Manual | 1 pc |



| | | | |
|--|--|--|-----------------------------|
| | Please be sure to read the instructions before use and follow as described. | | |
| | Do not apply it to the person as following: 1. Who has acute disease or bloody disease over 38°C 2. Who has allergy and hemophilia 3. Who has melanosis of the face 4. Who has high internal heat 5. Who regularly visits hospital 6. Who takes medicines 7. Who is under 12 years old | | |
| | Do not use it on the power other than AC 220V. It may cause fires or malfunction. | | |
| | Do not use it near the place with high temperature, dust or humidity (Near the humidifier). It may cause malfunction. | | |
| | Do not repair, disassemble or alter the controller and the body. It may cause malfunction. | | |
| | Do not touch the device or detach the power cords with wet hands. It may cause electric shock. | | |
| | Do not use the stripped cord or propped cord. It may cause fires(Be sure to replace it with another). | | |
| | Do not use the propped cord which has crack or scares. It may cause scares on the skin. | | |
| | When you don't use it for a long time, unplug it from the outlet. | | |
| | Be sure to have an inspection for test. If any troubles are found, get it repaired. | | |
| | No one other than the operator can touch it. It may cause accidents or malfunction. | | |
| | Alternating current | | Serial No |
| | Direct current | | Manufacturer |
| | Date of manufacture | | CE mark |
| | Attention, consult accompanying documents | | Consult instruction for use |

biopen[®]

User Manual