Lactation Massager

- EN -

User Manual
Model No.: EHL038

Healthcare-Manager.com

EHL038
V01: 20230105

User Manual (EN)	01-09
Benutzerhandbuch (DE)	10-19
Manuale d'Uso (IT)	20-29
Manuel d'utilisation (FR)	30-39
Manual de Usuario (ES)	40-49
Gebruikershandleiding (NL)	50-59

Contents

Overview	01
Prior to Use	02
How to use	02
What's in the box	05
Charging your massager	05
Specifications	06
Cleaning and storage	06
Environmental condition for normal working transport and storage	07
Important information	07
Warning	07
Warranty	08
Q&A	08
EMC information	08
Explanation of symbols	09

Overview

Easy@Home Lactation Massager EHL038 is a handheld lactation massager covered in soft silicone, which makes your breastfeeding experience more enjoyable. Eliminate clogged ducts, increase the speed of milk letdown and milk flow, and reduce discomfort from engorgement.

The massager has 10 vibration modes and 2 heating modes. It can relieve pain and discomfort and increase milk flow. You can also take it anywhere you go since it's so small and compact.

Additionally, it's waterproof, so you can use it in the shower (where the warm water may also help get the milk moving and clear the blockage).

Prior to Use

Before using your Lactation Massager, here are a few simple steps to ensure you have the best session.

1. Allow a few minutes for the massager to reach full heat. It is recommended to pre-heat the massager before pumping by using the heat only setting.
2. Explore different positions of the massager on your breast.
3. Wear a tight-fitting nursing/pumping bra to provide best compression. This massager will be optimized to provide constant and gentle heat throughout your breastfeeding or pumping session.

Note: Do NOT use if you have an allergy to silicone.

How to use

1. Product components:

- (1)  Vibration Mode Button


Press and hold for 3 seconds to turn on/off.
Press button quickly to switch vibration modes.

- (2)  Heating Mode Button

Press and hold for 3 seconds to turn on/turn off.
Press button quickly to switch heating modes.

2. Two heating modes:


Purple light means low heat and red light means high heat.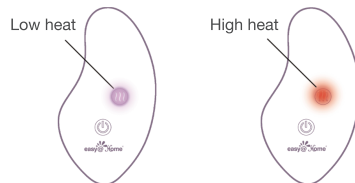


Note: The heating function will be turned off automatically after 30 minutes.

3. Diagram of vibration modes:

The below diagram shows ten vibration modes' impulse patterns. Press the vibration mode button briefly until you find the mode that works best for you.

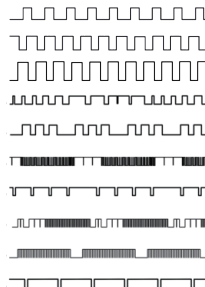4. Explanation picture of massager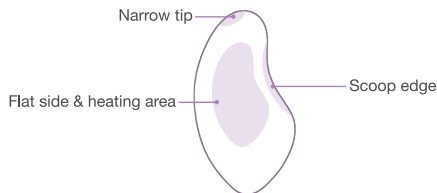

Procedure:

Step 1. Turn on the heating function. A quick rise in skin temperature will increase blood circulation to the targeted area, and increase stimulation for faster let-down and increased milk flow.

*** Note: Make sure to start with low heat to check your skin's sensitivity to heat and switch to high heat mode if necessary.**

Step 2. Press the vibration mode button to turn on massager then choose a comfortable vibration mode.

Step 3. Use the flat side of the massager to rub breast with appropriate pressure until you feel a little push, from around of breast toward the nipple. If the breastfeeding let-down does not work perfectly, you can use the scoop edge to massage different parts of breast. The scoop edge has a narrower contact area than flat side so it will provide a stronger stimulation and pressure to guide the milk.


Step 4. Use the narrow tip of the massager massaging clogged ducts during breast feeding or milk pumping.

NOTE:

To avoid burning skin, do not use heating mode for more than 10 minutes in one area. Keep moving the massager positions on your breast to avoid a single area from overheating. If the massager is too hot for skin, stop using immediately.

What's in the box

- 1* Lactation Massager
- 1* USB charging cord
- 1* Storage bag
- 1* Manual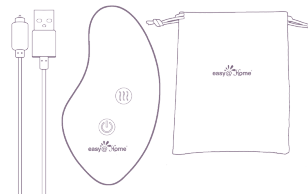


Charge your massager

1. The vibration mode button will blink a purple light when the battery needs to be charged, which indicates 10 minutes of battery life remain.
2. A bright purple steady light means fully charged. It takes 2.5-3 hours for a full charge.
3. Place the massager on a desk or flat area to make sure that the plug will not come loose when charging. Do not plug in too hard or the charging cord can be damaged.

NOTE:

1. Ensure the charging cord firmly attaches to the massager for charging.
2. To ensure the use time of the massager can reach 120 min or full capacity, please use after fully charged.

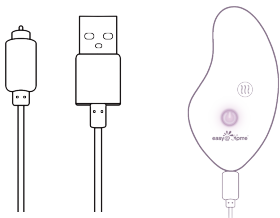Specifications

Usage Time: 120min for continuously using after battery fully charged

Size: 110 mm*60 mm*41mm

Battery Capacity: 750 mAh

Weight: 122g

Color: Purple

Modes: 10 vibration modes & 2 heat levels

Heating Temperature: 42°C~45°C (107.6°F~113°F)

Charging Electric Current: 370mA

Service Life: 3 years

Cleaning and storage

- ALWAYS clean your massager before and after every use.
- Unplug the charging cord before cleaning.
- Clean with mild soap and warm water. Do NOT submerge in water.
- Air dry naturally and avoid direct sunlight.
- Store in a dry and dust-free place.

Environmental condition for normal working transport and storage

- Normal working ambient temperature: 5°C ~ 40°C (41°F ~ 104°F)
- Normal working ambient humidity: 40% ~ 70%RH
- Transport and storage ambient temperature: -10°C ~ 65°C (14°F ~ 149°F)
- Transport and storage ambient humidity: 15% ~ 90%RH
- Atmospheric pressure: 70kPa - 102kPa

Important information

- Do not use while charging.
- Keep away from water when charging.(But it can be used in shower after fully charged.)
- Unplug the cable before cleaning.
- Do not touch plug with wet hands.
- Keep away from young children.
- Avoid using the massager if it is damaged.

Warning

Your safety is critical to us. However, burns may result from improper use. To reduce the risk of burn or any harm, this product must be used in accordance with the following instructions.

- Do not use while sleeping.
- Do not use on an infant.
- Do not use on skin (like back or abdomen) where you feel temperature insensitively or you will be burnt unconsciously if the massager's temperature is too hot.
- Do not use if the internal heating element is exposed.
- Start with low heat and pay attention to your skin sensitivity level. Also do not use in one area longer than 10 minutes at a time (as mentioned in the procedure portion of this manual).

Warranty

This product includes a 1-year warranty after the date of original purchase, against defects due to faulty manufacturing or materials. The warranty does not cover appearance abrasion caused by wear and tear or damage caused by accident, misuse, or negligence or theft. Please note, any attempt to open or take apart the device will void the warranty.

Q&A

Q1: Can the massager get wet if I want to use it in the shower?

A: You can operate the massager in the shower because it has IPX7 waterproof and covered in silicone. However, please do not submerge in liquid/water to maintain life span. Also, please dry the massager after use.

Q2: I have an allergy to silicone. Can I use the massager?

A: We highly suggest those who have allergy to silicone DO NOT use this massager. If you do get itching, swollen bumps or rashes over the skin after using this device for a silicone allergy or any adverse reaction, please visit a doctor as soon as possible.

EMC information

The product conforms to test standard(s):

EN IEC 55014-1:2021

EN IEC 55014-2:2021

BS EN IEC 55014-1:2021

BS EN IEC 55014-2:2021

Explanation of symbols

	Recyclable		Manufacturing date		Batch code
	Humidity limitation of 15% ~ 90%		Atmospheric pressure limitation of 70kPa ~ 102kPa		Temperature limit of -10°C ~ 65°C (14°F ~ 149°F)
	Keep out of reach of children		Manufacturer		UKCA marking of conformity
	CE marking of conformity		Authorized representative in the European Community		Authorized representative in the United Kingdom
	Refer to instruction manual		WEEE (Waste Electrical and Electronic Equipment)		

Healthcare-Manager.com

Questions or Comments?

E-mail: service@healthcare-manager.com

Easy Healthcare Corporation

360 Shore Dr., Burr Ridge, IL USA 60527

Made in China

Scan the code to get the manual in more languages

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