Ovulation Test INSTRUCTION FOR USE

[EN]

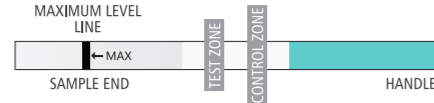
Please read all the information in this insert before performing the test.

[REF] ARS-LH

One Step Luteinizing Hormones (LH) Test
Luteinizing hormone (LH) Test

The Areta Ovulation Test Strip is an *in vitro* diagnostic test device intended for the rapid determination of human Luteinizing Hormones (LH) in urine to predict when there is a LH surge, and in turn, when you are likely to ovulate. You can easily carry out yourself.

- It is intended for self-testing use.
- For *in vitro* diagnostic use only.

1 HOW DOES IT WORK?

The LH (Luteinizing Hormone) which is in the urine of normal women will increase dramatically in the middle of the menstrual cycle. The LH increase triggers ovulation which is when the egg is released periodically from normal fertile women. WHO experts state that LH testing is a reliable way to detect ovulation. Ovulation will occur after 24-48 hours following a positive test.

The ovulation test contains antibodies which specifically react with LH. When the sample end of the test is dipped in the urine specimen, the urine specimen is applied to the strip and migrates along the strip. A colored line will appear at the Control Zone if the test has been performed properly. The control line also works as a reference line for you to find the LH peak time when the test line is as dark or darker than the control line.

2 CONTENT OF THE TEST KIT

- One pouch containing a reaction strip and a desiccant. The desiccant is for storage purposes only, and is not used in the test procedures.
- Leaflet with instructions for use.

3 WHAT ELSE DO YOU NEED?

- A clean, dry, plastic or glass container to collect the urine.
- Timer or clock.

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1. PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Do not use the test past the expiration date printed on the pouch.
- Do not use the kit if the pouch is punctured or not well sealed.
- Keep out of the reach of children.

5 STORAGE AND STABILITY

- Store at 4°C to 30°C (39°F to 86°F) in the sealed pouch up to the expiration date.
- Keep away from direct sunlight, moisture and heat.
- DO NOT FREEZE.
- You should perform the test soon after opening the pouch. You should keep the test in a dry environment before use.

6 WHEN TO START TESTING?

The length of the menstrual cycle is the duration from your first menstrual bleeding day to the day before the next bleeding begins. Determine the length of menstrual cycle before test. Please refer to the chart below to determine when you should start testing. If your cycle is shorter than 21 days or longer than 40 days, consult a physician.

If you do not know your cycle length, you may begin the test 12 days after your first period since the average cycle length is 28 days. Perform 1 test each day over a 5 days period, or until the LH surge has been detected.

Cycle Chart	Cycle Length	Day of Cycle to Begin Testing	Day	5
	21	days	day	5
	22	days	day	6
	23	days	day	7
	24	days	day	8
	25	days	day	9

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Cycle Chart

7 HOW TO COLLECT URINE?

Urine specimens may be collected in any clean, dry, plastic or glass container.

8 HOW TO DO THE TEST?

- The test strip and urine have to be at room temperature 15°C ~ 30°C (59°F ~ 86°F) for testing.
- Remove the test strip from the sealed pouch.
- Immerse the strip into the urine with the arrow pointing towards the urine. Take the strip out after 5 seconds and lay the strip flat on a clean, dry, non-absorbent surface (such as the mouth of the urine container).

IMPORTANT: Do not allow the urine level to exceed the MAX (marker line), otherwise the test will not perform correctly.

26	Tage	Tag	10
27	Tage	Tag	11
28	Tage	Tag	12
29	Tage	Tag	13
30	Tage	Tag	14
31	Tage	Tag	15
32	Tage	Tag	16
33	Tage	Tag	17
34	Tage	Tag	18
35	Tage	Tag	19
36	Tage	Tag	20
37	Tage	Tag	21
38	Tage	Tag	22
39	Tage	Tag	23
40	Tage	Tag	24

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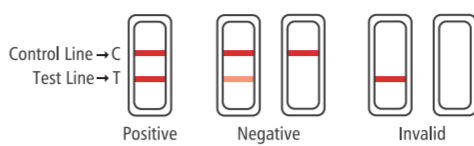
4. Read results in five minutes. *Do not read results after more than 5 minutes.*

9 HOW TO READ THE RESULTS?

- Positive (LH Surge):**
If two color lines are visible, and the test line is equal to or darker than the control line, one will probably ovulate in the next 24-48 hours. If trying to get pregnant, the best time to have intercourse is after 24 but before 48 hours.

- Negative (No LH Surge):**
Only one color line appears in the control zone, or the test line appears but is lighter than the control line. This means there is no LH surge.

- Invalid:**
No visible line at all, or there is a visible line only in the test zone and not in the control zone. Repeat with a new test kit.

10 UNDERSTANDING TEST RESULTS

- The test works only when the test procedures are precisely followed.
- This test may not be used as a form of birth control.
- Consult a doctor if irregular or unusually long cycles are experienced.

11 QUESTIONS & ANSWERS

- Do not know my usual cycle length. How will I know when to start testing? If you wish to start using the test before knowing your cycle length, we recommend you start testing on day 12.

- What time of day should I take the test? You may test any time of day, but you should test at approximately the time

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