Pregnancy Test

User Manual

-EN-

For Self-testing


REF PMS-110/PMS-140/PMS-150/
PMS-160/PMS-180/PMS-1100

INTENDED USE

Premom pregnancy test is an immunochromatographic assay designed for the rapid determination of human chorionic gonadotropin (hCG) in urine. The test is used to obtain a visual qualitative result. For *in vitro* self-testing use only.

PRINCIPLE

Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta shortly after the conception and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone. When the test is inserted to a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the Test Zone region of the membrane, it will form a colored line. Absence of this colored line suggests a negative result. To serve as a procedure control, a colored line will appear at the control zone region if the test has been performed properly.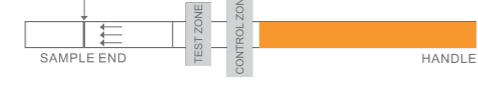

MATERIAL SUPPLIED

PMS-110/PMS-140/PMS-150/PMS-160/PMS-180/PMS-1100
10/40/50/60/80/100 Test strips (individually wrapped).
One pouch contains a test strip and a desiccant. The desiccant is for storage purposes only and is not used in the test procedures.

2. One instruction for use.

MATERIAL REQUIRED BUT NOT PROVIDED

- Clock or Timer
- Specimen collection containers.

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Do not use the test kit beyond expiry date.
- Do not use the kit if the pouch is punctured or is not well sealed.
- Keep out of the reach of children.
- Urine specimens may be infectious: Ensure proper handling and disposal of all used reaction devices as per the local statute.

STORAGE AND STABILITY

- Store at 2°C to 30°C in the sealed pouch up to the expiration date.

QUESTIONS AND ANSWERS

1. Q: How soon after I suspect that I am pregnant can I take the test?
A: You can test your urine as early as the first day you miss your period. You can perform the test anytime of the day; however, if you are pregnant, first morning urine contains the most pregnancy hormone.

2. Q: Do I have to test with first morning urine?
A: Although you can test any time of the day, your first morning urine specimen is usually the most concentrated of the day and would have the most hCG in it.

3. Q: How accurate is the test?
A: A clinical evaluation was conducted comparing the results obtained using the pregnancy test to another commercially available urine membrane hCG test. The consumer clinical trial included 216 urine specimens: both assays identified 76 positive and 140 negative results. The results demonstrated >99% overall accuracy of the pregnancy test when compared to the other urine membrane hCG test.

4. Q: Which factors may affect the test result?
A: Drugs which contain hCG (such as Pregnyl, Profasi, Pergonal, APL) can give a false positive result. Alcohol, oral contraceptives, painkillers, antibiotics or hormone therapies that do not contain hCG should not affect the test result.

5. Q: What should I do if the result shows that I am pregnant?
A: See your doctor to confirm that you are pregnant and to discuss the steps you should take.

HOW TO READ THE RESULTS?


2. DO NOT FREEZE.
3. The test should be used immediately after opening.

SPECIMEN COLLECTION

Any urine specimen is appropriate for pregnancy testing but the first morning urine specimen is optimal because of its highest concentration of hCG.

TEST PROCEDURE

- Remove the test strip from the sealed pouch.
- Dip the white absorbent tip of the test into the urine up to the black "MAX" line and keep the tip in urine until dye rises into the results window.
(Est. time: 5-10 seconds)
- Lay the test strip flat on a clean, dry, and non-absorbent surface.
- Read the result between 3 and 5 minutes after immersion. Do not read results after more than 5 minutes.
- The test strip and urine have to be at room temperature (15-30°C) for testing.

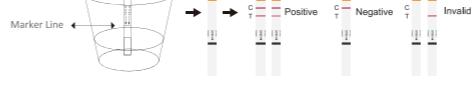NOTES

- Negative (not pregnant):**
Only one color band appears in the Control Zone and no Rosy Pink band on the Test Zone. This indicates that no hCG has been detected.
- Positive (pregnant):**
Distinct color bands appear in the Control and Test Zones. It indicates that you are pregnant. The color intensity of the test bands may vary since different stages of pregnancy have different concentrations of hCG hormone.
- Invalid:**
No visible band at all, or there is a visible band only in the Test Region and not in the Control Region. Repeat with a new test kit. If the test still fails, please contact us for assistance.
- Control line → C Test line → T**
- | | | | |
|----------|----------|----------|---------|
| Positive | Positive | Negative | Invalid |
| | | | |

NOTE: If the color in the Test Zone is very faint, it is recommended to repeat the test two days later.

LIMITATIONS

- As it is with any diagnostic procedure, a confirmed pregnancy diagnosis should only be made by a physician after evaluating all clinical and laboratory findings.
- If a urine sample is too dilute (ie, low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, another urine specimen should be collected 48 hours later and tested.
- Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
- Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used to diagnose pregnancy unless these conditions have been ruled out.
- A normal pregnancy can not be distinguished from an ectopic pregnancy based on hCG levels alone. Also, spontaneous miscarriage may cause confusion in interpreting test results.
- Sterility treatments, based on hCG, may cause false results. Consult your physician.

PERFORMANCE CHARACTERISTICS

A layer users evaluation was conducted comparing the results obtained using the Pregnancy Test (Urine) to professional users. The study included 60 urine specimens, and layer users and professional users

Keep dry

Keep away from sunlight

Recyclable

If you have any questions, please contact us via email support@premom.com

Distributed by Easy Healthcare Corporation

360 Shore Dr., Burr Ridge, IL USA 60527

To learn more, please visit us at www.premom.com

Scan the code to get the manual in more languages

MEANING OF SYMBOLS ON PACKAGE

	Consult instructions for use
	Store between 2-30°C
	Lot number
	CE marking of conformity
	For <i>in vitro</i> diagnostic use only
	Tests per kit
	Authorized representative
	Manufacturer
	Used-by date
	Catalog#
	Do not reuse

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