



**Q1: There is a lot of difference between the control lines of two different tests. Is this a concern?**

No. Variations in the color of the control line will not affect the test result.

**Q2: Can test results be interpreted after more than five minutes?**

No. Test results must be read in 5 minutes. Though a positive result should not change for several days, a negative test result may change to a false positive within minutes after the end of the testing period, which would not be an accurate reading. It is always best to read the results at the 5 minute testing period and then discard the test to avoid confusion.

**Q3: I have waited longer than 5 minutes and still there are no lines in the result window. What does this mean?**

There should always be a control line in the result window at 5 minutes for the test to be valid. If not, the test is invalid. The invalid results are normally caused by three major reasons: (1) not placing the absorbent tip in urine for the full 10 seconds, (2) urine is insufficient, or (3) urine passed the arrow line on the test. You should retest with another test and make sure to place the absorbent tip in urine for 10 seconds meanwhile make sure you have enough urine while avoiding urinating past the arrow line on the pad. If you still see the invalid result, contact our customer service team.

**Q4: What is the sensitivity level (mIU/mL) of the midstream test?**

The Premom midstream pregnancy test can detect human Chorionic Gonadotrophin (hCG) in urine at the FDA recommended standard of 25 mIU/mL.

**Q5: Can I dip the pregnancy test into liquid other than urine?**

No. Premom hCG Urine Test is a rapid pregnancy test, it detects the presence of human Chorionic Gonadotrophin (hCG), which appears in urine very early during pregnancy. Please just followed manufacturer' s instructions, do not test any other liquid if you want to get the correct test result.

**Q6: What is the differences between the Premom pregnancy midstream and strip tests?**

Premom pregnancy test strips and midstream tests are the same and both are used to detect pregnancy. The difference between using a midstream test over a strip test is that you can urinate directly onto the mainstream' s absorbent tip for direct urine testing. No need to use a disposable urine cup to collect your urine sample.

**Q7: What is included in the product package?**

Premom pregnancy tests contains 3 hCG midstream tests and 1 user manual.

**Q8: Is this pregnancy test accurate?**

Yes, our tests our 99% accurate if the test is done properly. Please refer to the user manual before testing. We recommend using first morning urine when performing the

test as this is when hCG concentrations are the highest.

**Q9: How do I take the test and how long should I wait to read my test result?**

Place the absorbent tip in the urine stream for 10 seconds to be thoroughly wet. Re-cap the test and wait for the control line to appear. Positive results may be observed as soon as 1 minute. However, the complete reaction time of 3 minutes is required. User should read results between 3 and 5 minutes. Do not read results after 5 minutes as after 5 minutes the results may lead to false positives.

**Q10: Do two lines mean a positive result?**

In most situations, yes. Any color of a test line appearing means the test is positive. Under certain conditions, the test may give a false positive result. If a urine sample is too diluted, it may not contain a representative level of hCG in your urine for the test to pick up. If pregnancy is still suspected, test again with another urine specimen in 48 hours.